## Shower Chairs & Seats

## Fixing and Maintenance Instructions

**N.B. This equipment must be installed by a competent person and a risk assessment may be needed as to the suitability of the chair for the particular user.**

### **MODELS AND WEIGHT LIMITS**

VB543	Solo Compact Shower Seat	115 kg (18 st.)
VB633	Solo Cuxton Shower Seat	115 kg (18 st.)
VB548	Hunton Standard Shower Seat	190 kg (30 st.)
VB544/VB544W	Solo Compact Shower Seat with Leg	159 kg (25 st.)
VB535	Solo Deluxe Shower Chair (Padded)	190 kg (30 st.)
VB539	Solo Deluxe Shower Chair	190 kg (30 st.)
VB542/VB542R	Solo Compact Shower Seat Padded	115 kg (18 st.)
VB634	Solo Standard Shower Chair Padded	190 kg (30 st.)
VB546	Hunton Deluxe Shower Seat	190 kg (30 st.)
VB532	Solo Standard Shower Seat	190 kg (30 st.)
VB629	Solo Cuxton Shower Seat	159 kg (25 st.)
VB629	Solo Cuxton Shower Seat	159 kg (25 st.)
VB651	Duo Deluxe 2 in 1 Shower Seat	190 kg (30 st.)

**DO NOT exceed the weight limit stated – doing so could put the user at risk.**

### **ALL SHOWER SEATS**

Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the walking stick.

Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

### **CLEANING**

All of the product components are manufactured from quality materials. However, it is very important that you only clean your shower chairs and stools using a nonabrasive cleaner or mild detergent with a soft cloth. Avoid using lime scale remover. Abrasive cleaners e.g. AJAX and or abrasive cleaning pads will seriously damage your product beyond repair.

Certain products have clip on/clip off backs and/or seats; you can easily remove these if required, clean with a mild detergent and soft cloth in a warm water solution. Do not exceed 50°C when cleaning the seat cover and/or back as this will permanently damage the plastic.

**N.B. Regular daily cleaning will reduce the build-up of limescale on your product.**

UNDER NO CIRCUMSTANCES use any abrasive cleaners or materials on any of the components. Also check that the cleaner you intend to use is suitable for ABS, Vinyl, Polymer Plastics and Thermo plastic rubbers and Polyurethane Rubber.


## USE OF BATH OILS

Caution should be observed if you are using any form of bath oils or oily skin preparation with your shower chair as this may impair your natural contact skin resistance.

In the event of using any such oil, your seat must be thoroughly cleaned with a suitable detergent to break down any oily residue. Failure to observe this recommendation could impair your safety.

## CARE & MAINTENANCE

Please make a safety check of the product at regular intervals (at least every 6 months) or on reissue, or if you have any concerns.

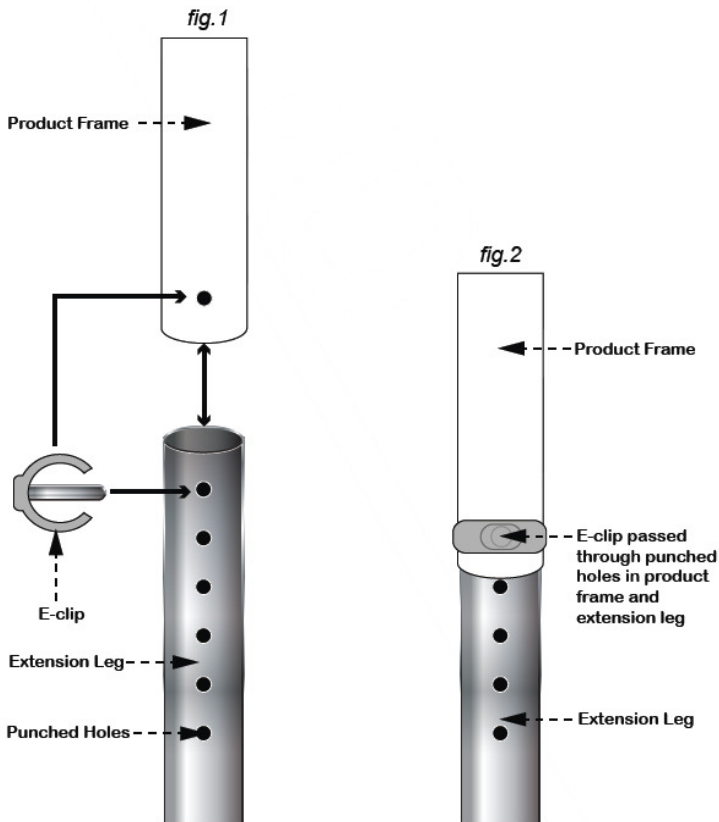## **ADJUSTMENTS TO EXTENSION LEGS (IF APPLICABLE)**

Please adjust the legs prior to installation. Remove the 'E' clip from the extension leg and extend all four legs evenly to the desired height ensuring you do not exceed the last punched hole of the extension leg. (see fig. 1)

Once you have achieved the desired height; reinstall the 'E' clip, ensuring that it has passed through the extension leg and is fitting neatly and snug around the outside leg. (see fig. 2).

Never install the extension legs unevenly or slanting, this will compromise the safety of the user.

## **FIXING THE SHOWER SEAT**

Choose a suitable fixing for the substrate that you intend to fix the shower seat to and secure the back plate of the shower seat to the substrate. Make sure that the fixing is secure before allowing it to be used.

## **TESTING THE SHOWER SEAT**

Once you have installed the shower seat check your installation and the movement of the shower seat by raising the set from the down position to the up position.

- Check that there are no obstruction
- Check that the movement of the seat is smooth and unhindered
- Check that the seat legs (if applicable) are sitting evenly on the floor

## **RAISING AND LOWERING THE SHOWER SEAT**

To raise your seat, simply lift the front of the seat and the legs will automatically fold inwards towards the wall. Reverse the action to lower.

## **FOLDING ACTION**

If required, your shower seat can be folded up when not in use. The seat is held in position by friction only.

Once you have installed the seat, check that there is sufficient friction to keep the seat in the upright position when required.

If the seat starts to descend, then you will need to tighten the bolts either side of the seat hinges. This operation should be sufficient to maintain the seat in the closed position.

Additionally, it will be necessary for the bolts to be tightened periodically, if the seat starts to open on its own accord.

## **THREADED LEG ADJUSTMENT**

Your shower seat may have special threaded leg adjustment as opposed to the standard extension leg.

To adjust the height of your seat, carefully open the stainless steel M10 locking nut and undo the threaded stem to the desired height. On no account exceed a finished height of 470 mm measured from the top of the seat to the bottom of the black circular foot, as this may compromise the safety of the user.

Once you have reached the desired height, lock the stem in place by tightening the lock nut against the black indented bush.

**NB. Occasionally the extension stems may be slightly angled inwards to the frame. This is not a fault and will not impair the function or safety of the product, provided that you tighten off the lock nut as required.**

## **REISSUE**

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

This includes:

- PAT test for electrical items
- Tightness of all nuts/bolts/castors
- Other screw in/bolt in/push in components.
- Also check all upholstery for security, splits, etc.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

When reissuing, a risk assessment may be required as to the suitability of the chair for a particular user.

## **IMPORTANT INFORMATION**

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

**Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX**  
**Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: [www.aidapt.co.uk](http://www.aidapt.co.uk)**  
**Email: [accounts@aidapt.co.uk](mailto:accounts@aidapt.co.uk) • [adaptations@aidapt.co.uk](mailto:adaptations@aidapt.co.uk) • [sales@aidapt.co.uk](mailto:sales@aidapt.co.uk)**

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**Altai Germany (Altai Europe Ltd.)**