



# Onyx Seat with adjustable leg

## INSTALLATION INSTRUCTIONS

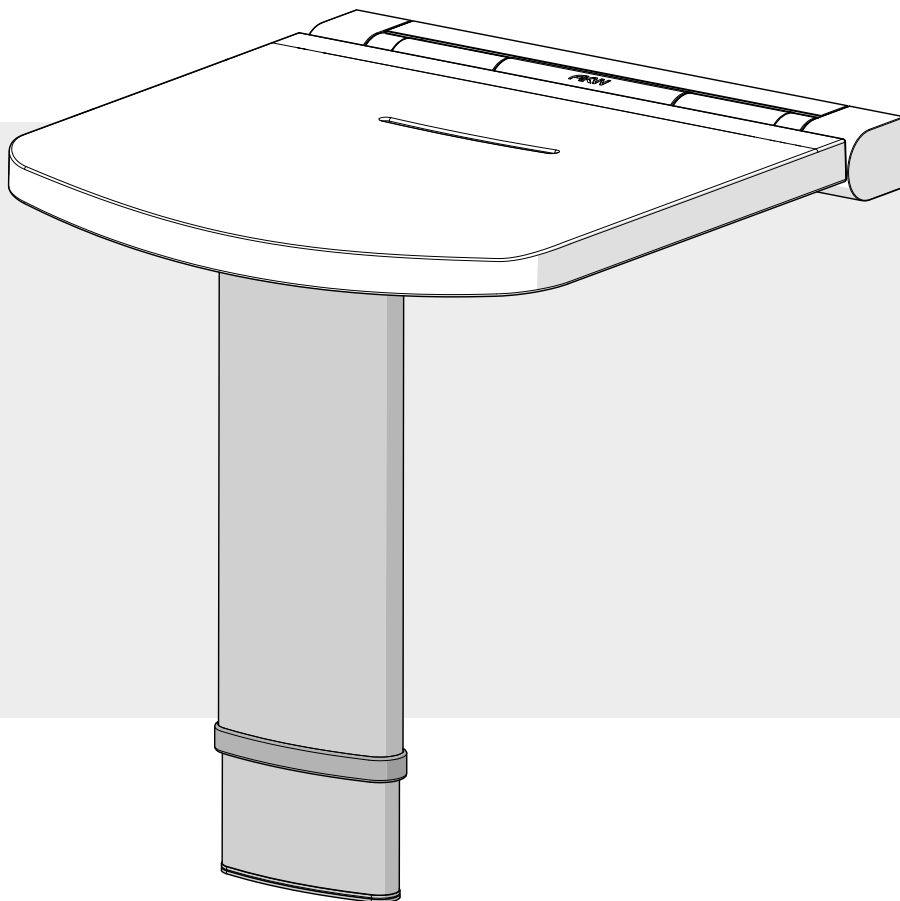

Please read all instructions before installation and leave this document with the end user for future reference as it contains important warranty information

## Dimensions

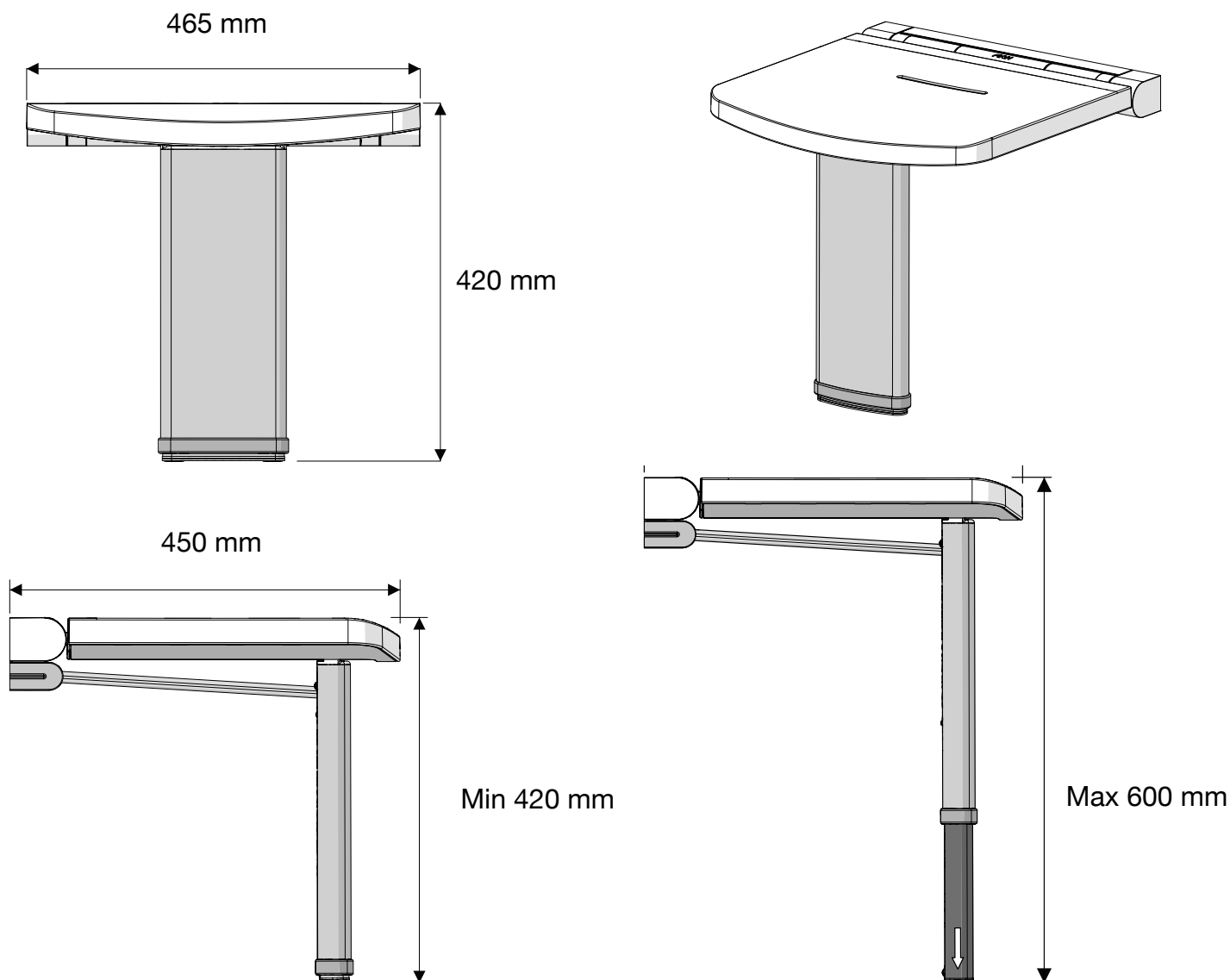


### Before you start


Please remove all protective covering and examine carefully. Check that all items are present before discarding of any packaging. Please notify the retailer immediately of any damage or faults. The manufacturer cannot be held responsible for any installation costs that may arise from damaged or faulty products being installed and are not in accordance with these instructions. The guarantee does NOT cover impact related damages, see warranty details.

### Safety


- **Max 200kg load rating** (depending on the wall type, use suitable fixings) The maximum load rating for this seat is subject to the product being correctly fitted to a suitable wall/ mounting surface and the product being used on a suitable shower tray or floor surface. Acrylic shower trays are not suitable for use with this product. Ensure the wall type is suitable for this installation.
- Do not stand on this seat.
- Always sit in the middle.
- Children should be supervised at all times when using this seat.
- Keep children away from small parts and packaging.
- Please dispose all packaging in accordance with the local recycling authorities.

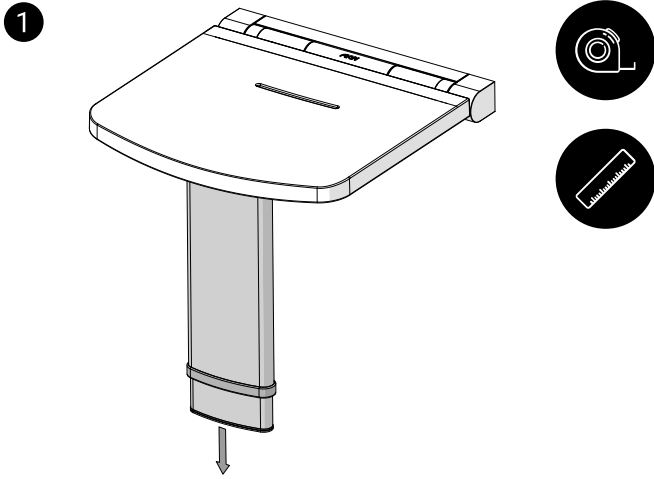


**1.** Determine the user height of the seat and measure this height against the wall. Check for any obstacles or pipework before any drilling or fixing.

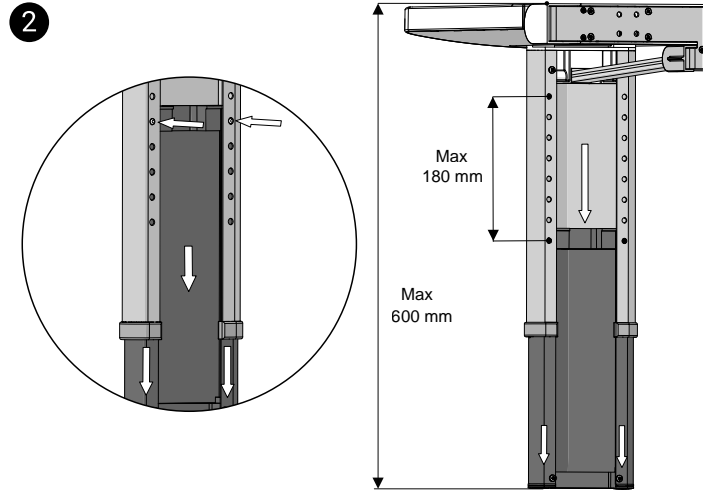


**2.** Adjust the leg height by pushing in the leg pins and slide the adjustable leg downwards. Always ensure that the leg pins are locked into position and the leg height cannot move before the user sits down.

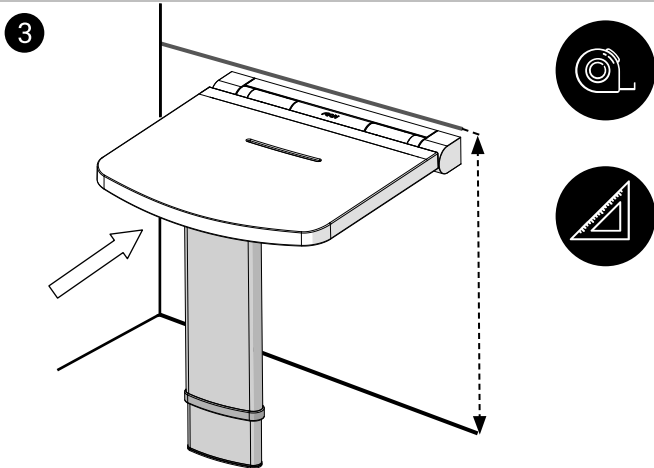


**3.** Place the seat against the wall and mark the height.

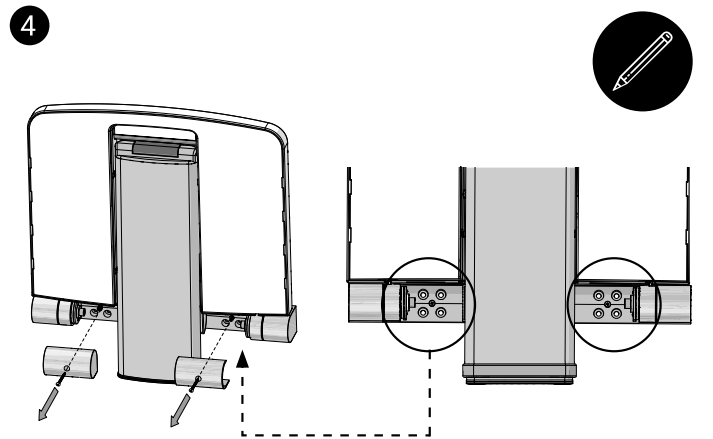

**4.** Raise the seat to remove the bracket covers to get to the mounting holes. Using a level and a pen, mark the hole positions, making sure that the seat is horizontal.

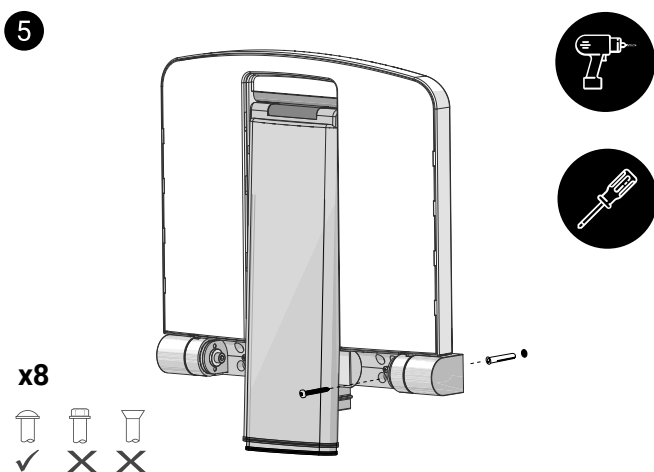

**5.** Drill the holes and insert the appropriate fittings. (Fittings not supplied) Do not use countersunk heads or over-tighten the screws.

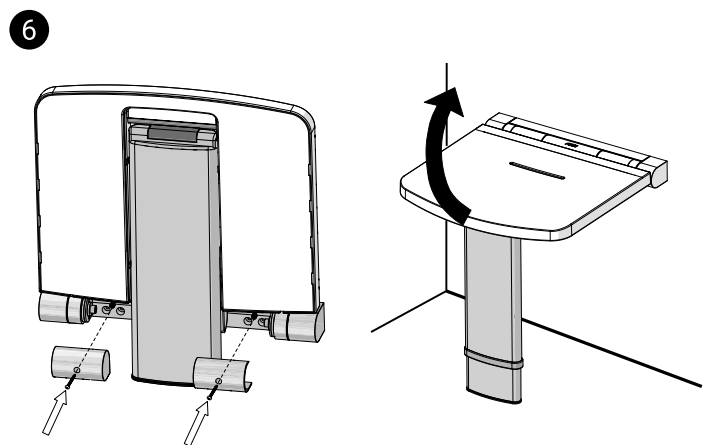

**6.** Position the seat over the holes and insert appropriate fittings and secure in place. Replace the bracket covers and test the seat operation.

# Contact Us

**AKW**  
Life Made Better

## Orders & Quotes

01905 823 299  
orders@akw-ltd.co.uk

## General Enquiries

01905 823 298  
sales@akw-ltd.co.uk

## Technical Enquiries

01905 560 219  
technicalenquiries@akw-ltd.co.uk

## Kitchen Enquiries

01905 823 262  
kitchens@akw-ltd.co.uk

## Fax

01905 823 297

## AKW

Pointon Way, Hampton Lovett  
Droitwich Spa, WR9 0LR

[www.akw-ltd.co.uk](http://www.akw-ltd.co.uk)  
[orders.akw-ltd.co.uk](mailto:orders@akw-ltd.co.uk)

## Follow us on Social:



## Guarantee

Life time guarantee only applies to manufacturing or material defects. This is conditional on the one-time correct installation of the product and does not apply to breakages, inappropriate use or accidental damage. This guarantee does not cover damage caused by the lack of maintenance including the build up of grime, dirt, lime-scale and waterborne debris. This does not cover damage resulting from inappropriate cleaning or water ingress, damage from water freezing or extreme temperature changes. Damage or defects that result from unintended use or modifications undertaken by none users. This guarantee excludes general wear and tear through usage and does not apply to surface finishes. This product should be cleaned regularly. Visit the website for more information.

## What to do if something goes wrong?

In the event that you should encounter a problem with this product, please contact your local installer and if this does not resolve the issue, then contact AKW technical team who will provide further advice and, if necessary, arrange for a maintenance engineer to visit and report any issues. See the T&Cs on our website or our catalogue for further information. None of the foregoing affects your statutory rights.