Ropox Toiletstøtte Straight Leg, 40-40120

/ Ropox Toilet Support Arms Straight Leg

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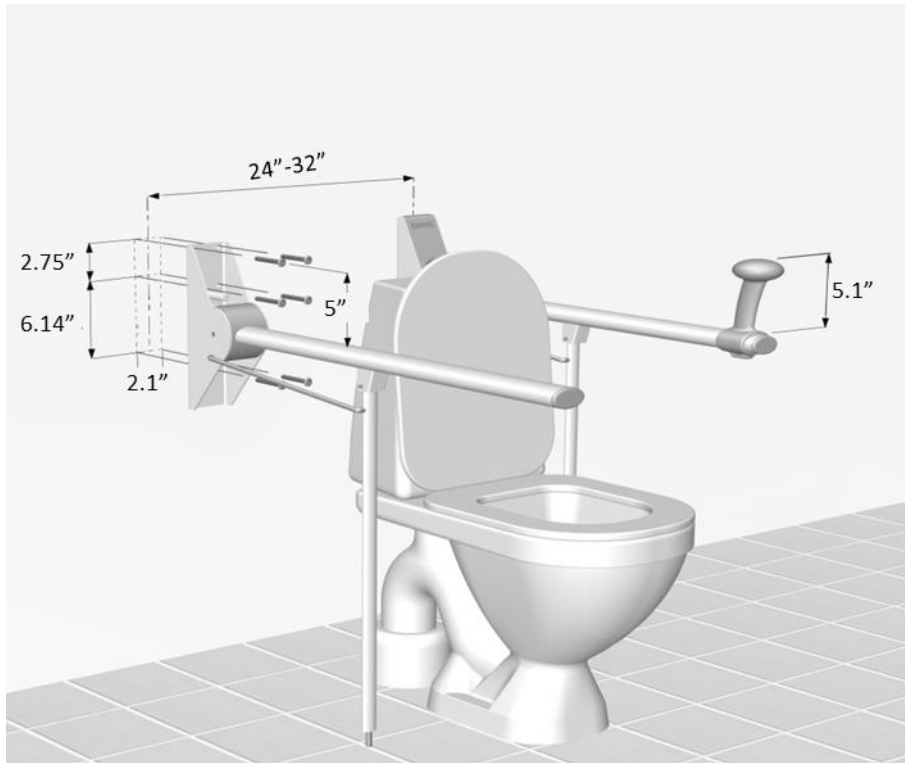
Please check with the user for individual demands which could affect the mounting position. Also make sure that national and/or individual state statutes are fulfilled


Please find attached mounting template showing position of the 6 holes for easy installation
User heights and installation heights (upper holes in mounting template/bracket)

User height (recommendation)	Installation height (upper holes)	Situation
30.7in (top of support arm)	35.75in	Ropox Straight Toilet Support Arm
26.4in (top of support arm) 31.5in (top of Get up/sit down)	31.4in	Ropox Straight Toilet Support Arm + Get up/sit down fitting
<p>Possible user height: 25.6in to 30.7in from floor to top side of arm support. If any specific user height is required, simply position the 2 upper holes of the bracket (or the mounting template) 5in above the required height of the support arm. <u>Example:</u> required user height is 30.5in Installation height = 30.5in+5in= 35.5in</p>		

Always read and understand the mounting instruction and safety in use before mounting this product.

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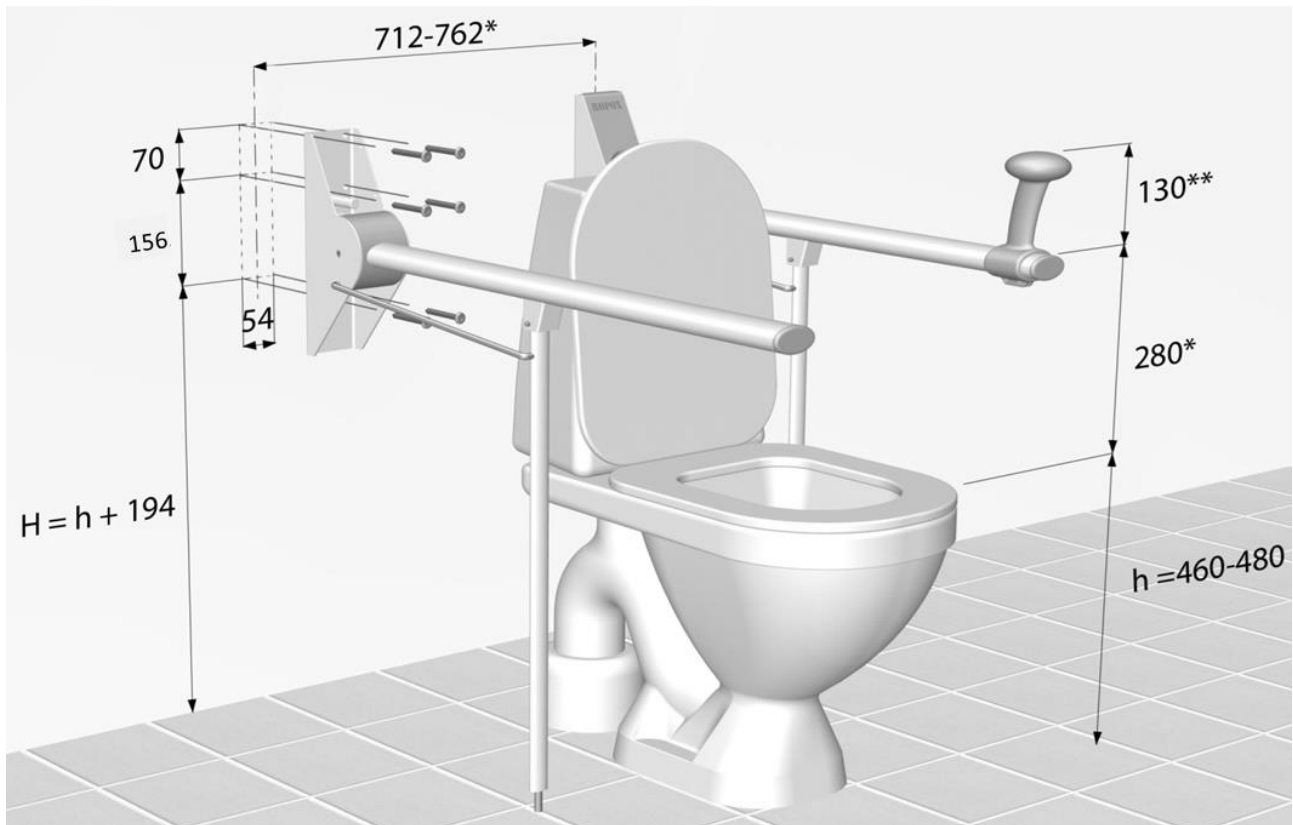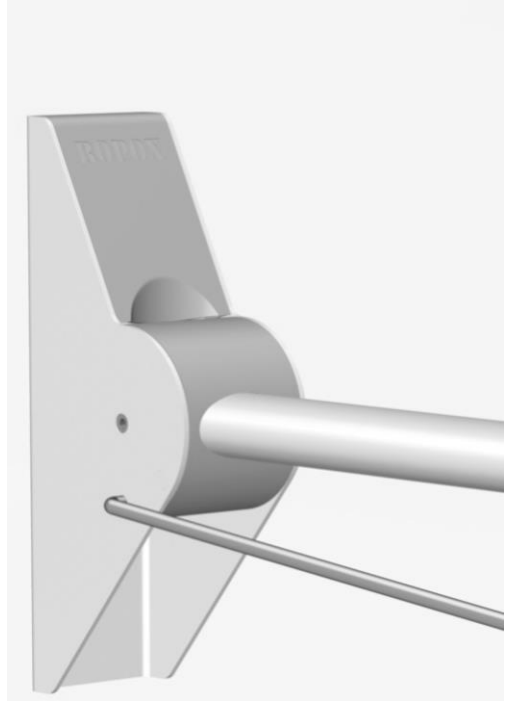
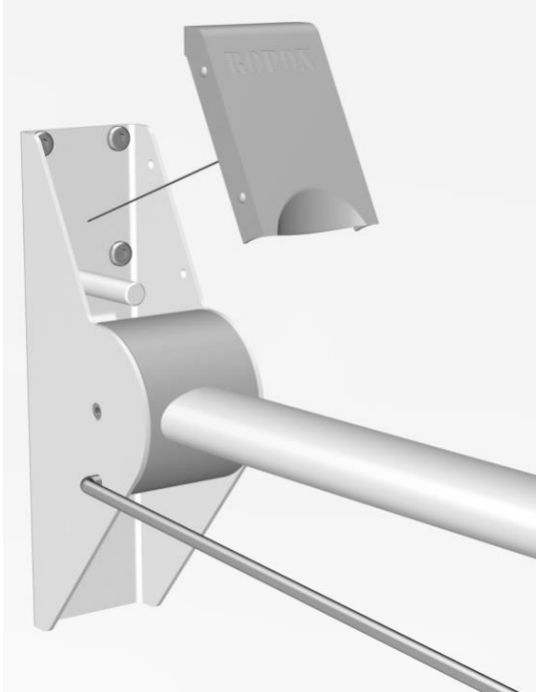
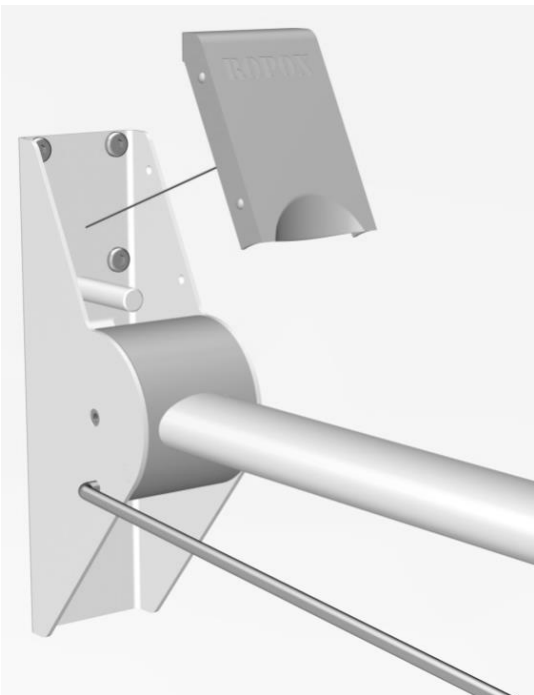
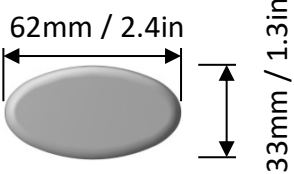
Safety in use

- Do not operate the toilet support in case of defects or damage.
- Do not overload the toilet support.
- Modifications, which may influence the operation or construction of the toilet support, are not allowed.
- Installation, service and repairs must be carried out by competent personnel.
- If the toilet support has not been assembled according to these mounting instructions the right to complain may become void.
- Only use Ropox original spare parts as replacement parts. If other spare parts are used, the right to complain may become void.

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Wall mounting

The screws and rawlplugs included are for mounting on a concrete wall. The installer must always examine the material, condition and strength of the wall and use appropriate screws and rawlplugs.

To comply with DS/ISO 17966 the maximum user load is set to 390kg/297lbs.

Maximum pull in the top screws are 90kg/220lbs.

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Maintenance

The toilet supports are maintenance free. For safety reasons it is recommendable to check all mounting screws and moving parts to be sure all is tightened ok, and that all attachment to the wall is in correct and safe condition.

Cleaning

To clean this product, do not use a high-pressure washer. This product is not machine washable. Simply use a soft cloth or sponge with soapy water (pH 7) of max 60°C. Always clean this product after use to prevent dirt from sticking to the surface.

Important:

Do not use scouring powder or other abrasives as they may scratch the surface. Also avoid the use of cleaning utensils with sharp edges. Never use strong mineral acids or alkali.

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CE-erklæring

Ropox hereby declares that the following product

40-40120 Ropox Toilet Support Arms Straight Leg

is classified as Class I and conforms to the following directive and standard:

Directives

Directive No. 93/42/EEC concerning Medical Devices (risk class 1)

Standards

DS/ISO 17966: 2016	Assistive products for personal hygiene that support users
DS/EN 14971: 2012	Medical devices - Application of risk management to medical devices
DS/EN 9999: 2016	Assistive devices for persons with functional disabilities - Classification and terminology

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Se Generelle salgs- og leveringsbetingelser på www.ropox.dk

Complaints

See General Terms of Sale and Delivery on www.ropox.com

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ROPOX A/S