Poids maximum utilisateur / Max. user weight / Maximumgewicht gebruiker / Peso máximo utilizador / Peso max. Utente/Peso máximo do utilizador / Maximales Benutzergewicht / Макс. вес пользователя.  
 Poids du produit / Product's weight / Productgewicht / Peso del producto / Peso del prodotto / Peso do produto / Gewicht des Produktes / Вес изделия

Référence P / Reference P / Referentie P / Referencia P / Codice P / Referência P / Referenz P / Код P :		048610 048710
Fonction Function Functie Función Funzione Função Funktion Назначение	Permet à l'utilisateur de prendre appui en toute sécurité. Allows a safety leaning. Zo kan de gebruiker in alle veiligheid ondersteund worden. Permite de apoyarse con seguridad. Permette di prendere appoggio con sicurezza. Permite ao utilizador apoiar-se em toda a segurança. Zum sicheren Abstützen des Benutzers. Служит для пользователя надежной опорой	
Descriptif Description Beschrijving Descripción Descrizione Descrição Beschreibung Описание		150 kg 0,800 kg
Numéro notice / Instruction sheet No. / Fichenummer / Número del manual / Numero manuale / Número do manual / Gebrauchsanweisungsnummer / Номер инструкции		A04861000
Date de création / Creation date / Aanmaakdatum / Fecha de creación / Data di creazione / Data de criação / Erstellungsdatum / Дата создания		22/12/2014
Révision n° / Revision No. / Revisie nr. / Revisión n° / Revisione n° / Revisão n° / Revisionsnr. / № редакции		02
Date / Date / Datum / Fecha / Data / Data / Datum / Дата		25/08/21

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**Do not install this equipment before reading the following instructions:**

**Fitting instructions.** The hinged bar has to be fitted so that the upper part of the bar (in horizontal position) is to 80 cm of the ground. To guarantee

P = PELLET ASC

the good use of this product, we recommend to fit it according to instructions.  
**Care and maintenance. IMPORTANT:** In order to guarantee the lasting quality of this product and its safety for the user, we recommend that when cleaning it you should use mild soap solutions.

**CAUTION: DO NOT USE** cleaning products made of: etones, hydrocarbons, acids, bases, esters and ethers. After use of a cleaning product, make a rinsing in the clear water.

**Safety warnings.** The support prop adds security to the user if it is fitted according to instructions. To avoid any pinching risks, do not put fingers in the mobile parts. **The product does not contain any hazardous substances. The manufacturer shall have no liability for any injury caused by improper installation or use of this support prop.**

At the end of its life cycle, the user must have the product recycled by a certified facility in the country of use.

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