

# doorline-variwedge™ Small\* (DVWS)


\*3 Sizes available; small, medium and large for thresholds up to 10cm (4inches)



**Before use:** Remove & retain all packaging • Read the instructions fully before using ramps. • Ensure the surface is firm, stable and able to bear the intended load • The capacity of the ramp is 300kg (47st). ⚠ Note: The ramps may have sharp edges. Use gloves while assembling

**Check and familiarise yourself with the contents:** 1x Variwedge Small ramp, 1x Variwedge Small leg, 3x Variwedge flush bolts, 3x Variwedge Flange nuts, 1x 10mm spanner, 1 length Foam Rubber self-adhesive strip

Contents:

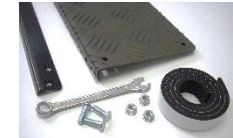

## Set up

- Measure the height of the threshold & choose the suited configuration (fig 1.)
  - Please note, the leg is not used for the lowest setting. Retain with fixings for future re-use/adjustment
- Assemble VariWedge ramp and leg as required
  - Insert flush bolt into relevant hole at one end of the leg (fig 2) (The mid setting uses the lower and the highest setting uses the upper flush bolt holes in the leg)
  - Insert the protruding bolt into the corresponding hole in the back face of the ramp (fig 3.) Screw the flange nut onto this bolt and hand tighten (fig 4.)
  - Insert the midway and end flush bolts through the corresponding holes (ensuring ramp & leg are parallel), add and hand tighten the flange nut. Tighten all 3 nuts with the spanner provided
- Position/fix the ramp as required
  - The foam rubber strip may be applied to the underside of the ramp to grip and protect the floor surface. Ensure the ramp surface is clean prior to application
  - There are 4 holes in the surface of the ramp for fixing as required (fixings not supplied)
  - The ramp has 6 notches that can be used to support the ramp from correctly spaced screws in a solid threshold

**Maintenance:** If the ramps become damaged or slippery in any way they should not be used. The ramps may be cleaned with normal cleaning products

**Recycling:** If you no longer require this product please liaise with a metals recycling contractor. If this product was loaned to you by a local authority, please contact them to discuss its removal from your property.

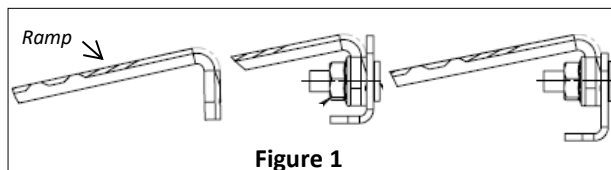


Figure 1

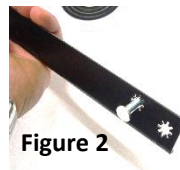


Figure 2



Figure 3



Figure 4

WARRANTY VOID IF LABEL REMOVED OR ILLEGIABLE. UK Patent Application No. 1404579.3 & (European) Registered Community Design No: 002434290/2/5/8. RampCentre is a registered trademark and Doorline & Variwedge are a trademarks of Enable Access AL9 SSD. We undertake a policy of continual review and research, which may lead to changes in product appearance and specification. **For technical support or details on your nearest stockist please call 0870 27 99 4 99**