




rampCENTRE[®]

permaramp-entry flap[™]

ultimate semi-permanent PVC threshold solution

Installation instructions

⚠ read carefully before installation/use ⚠

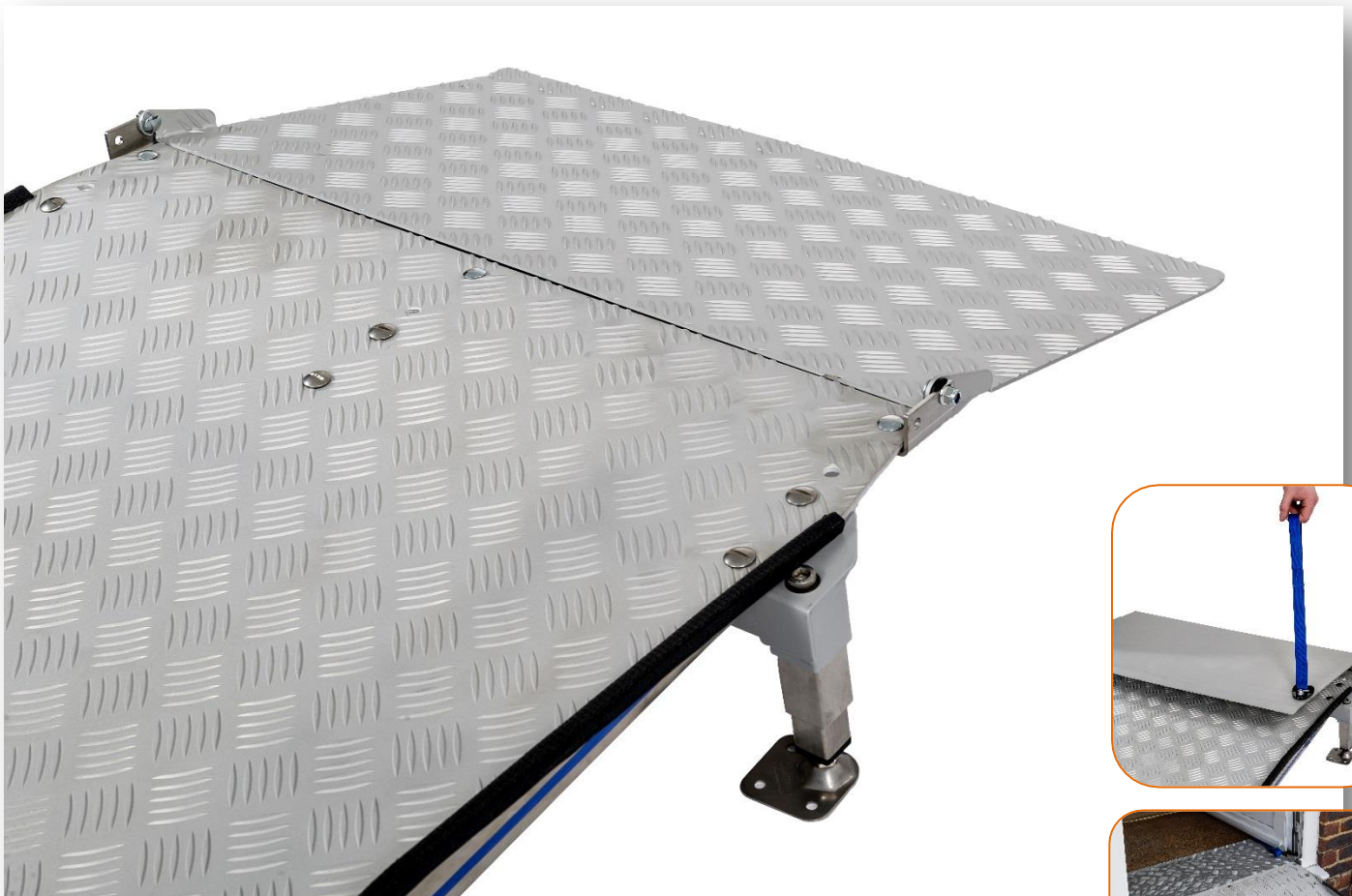

Designed &
Manufactured in the UK.



For technical support or details of your nearest stockist please call 0870 27 99 4 99

Before you start...

- Remove all packaging, check contents and ensure the products have not been damaged in transit.
- Retain packaging.
- Carefully read all instructions and warning texts
- Keep instructions for reference.

Warranty

- The products carry a comprehensive 2 year warranty for normal use, conditions apply.
- Use other than published/intended will void the warranty.
- Any modification/repair other than by manufacturer will void the warranty.
- Products will not be repaired/replaced if the serial number plaque has been removed/replaced.
- Proof of purchase will be required.



Safety

Observe product weights and use appropriate correct manual handling techniques and equipment. Be aware of pinch points between moving parts



= safety information

Contents

Assembled Permaramp Flap

Foam tape

10mm spanner

Installation

Note: Max inside depends on slope of outside ramp. I.E the gentler the slope outside = greater inside height. As a guide, 1:12 ramp slope outside = Approx 10cm (4") height inside

Install so top lip of permaramp is as close to threshold as possible ensuring installation does not interfere with door operation

Ensure the flap folds fully down onto the Peraramp

Apply foam tape to flap if required to protect inside floor surface

Strap handle can be fitted either side of flap