



WELCOME®

ECO FRIENDLY RAMP SYSTEM

Care & Safety Instructions

Before use

- Remove all packaging and ensure the ramps have not been damaged in transit. Retain packaging.
- Carefully read the instructions and warning texts before using the ramps
- Keep instructions for reference

About

- 'Gripdeck' surface is a highly effective access ramp surface that has been proven in use in both residential and public applications throughout the UK and beyond for many years.
- 'Gripdeck' was pioneered and developed by Enable Access working closely with the Building Research Establishment (BRE) in 2001/2 and then subsequently application tested for use and prolonged exposure at the BRE Research Site on top of a hill in East Kilbride.
- Slip tests for the Gripdeck® surface confirm that the surface grip is highly effective in protecting against the risk of a slip injury. Grip performance of (walking) surfaces is generally categorised as either; 'Dangerous', 'Marginal' or 'Safe'. The test results confirmed the Gripdeck® surface as 'Safe' by a large margin.
- Use in bare feet is not recommended as in different weather conditions the ramp surface can become very hot or very cold.

Maintenance

- All fixings and fittings should be regularly checked to ensure they have not become dislodged or loose in use and are secure and safe.
- Check regularly the ramp is in good condition.
- If the ramps become damaged or slippery or any fixings and fittings become loose in any way they should not be used (contact supplier/manufacturer).
- The ramps may be cleaned with normal cleaning products and left to dry before use.
- Maximum recommended cleaning temperature 50°C, please enquire for further details on use with industrial cleaning machines
- If you no longer require this product please liaise with a metals recycling contractor. If this product was loaned to you by a local authority, please contact them to discuss its removal from your property.

Warranty

- The ramps carry a comprehensive 2 year warranty for normal use, conditions apply.
- Use other than published/intended will void the warranty.
- Any modification/repair other than by manufacturer will void the warranty.



Designed & Manufactured
in the UK.

Ramp Centre is a registered trademark of Enable Access 16 Plantagenet Road Barnet EN5 5JG. We undertake a policy of continual review and research, which may lead to changes in product appearance and specification.

For technical support or details on your nearest stockist please call 0870 27 99 4 99