

Limitation of stroke length with memory-table-control and memory-adapter-cable

1. Unplug the control button (up and down) and plug in the memory-table-control.
2. Run down the motors to the lowest position
3. Press and hold the down-button for 5 sec until the control box clicks, the system is now set to zero
4. Run to the wanted minimum position (if it is at the bottom, go on to paragraph 5)
5. Click "S" and then "1"
6. Run to the wanted highest position
7. Click "S" and then "2"
8. Remove the memory-table-control
9. Insert the memory-adapter-cable (with yellow contacts)
10. Insert the normal control-button in the memory-adapter-cable. The system will now go between the two programmed memory positions. Down-button goes to memory position 1 and up-button to memory position 2.

If you have any questions please contact:

Independent 4 Life's Technical Department - 02392 755992

