

EN Operating / Care instructions Basicline 415

1. Operating instructions

Basicline 415 is a motorized height adjustable washbasin lift that is mounted under the mirror or mirror cabinet.

- Press and hold the button to start the operation.
- Release the button when the washbasin is at the desired level.

Pos. 1

Cable hand control

Pos 2.

Safety stop switch - prevents the descent of the lift in the event of an obstruction.



2. Maintenance

The Washbasin lift is maintenance-free. Greasing and other actions are made during the manufacturing for the lifetime of the lift. From safety point of view however some components shall be inspected each year.

- Inspections, service and repair work shall be made by competent persons.
- Remove any objects from the washbasin before maintenance and repair works are made.

3. Functional test

- Run the washbasin lift down and up all the way to respectively end position. Check that it moves freely, without hindrance and jarring sounds.
- Check that no cables or hoses are jammed and that the water and sewer hoses are properly fixed to the wall.
- Run the washbasin lift to the end position and make sure the hoses are long enough above the fixing point.
- Make sure the wall installation, washbasin installation and mirror installation are intact. If the frame is inclined due to wall installation, adjustments must be made.
- Check the function of safety switch:
Run down the lift while lifting the front edge of the wash basin. Make sure that the lowering is stopped.

4. Cleaning

Clean with lukewarm water and a non-scratching detergent containing soap or similar. Use a soft cloth or similar. After cleaning the surfaces shall be dried to avoid lime deposits.

Scratching detergents or tools e.g. steel wool may not be used.
Consider that the product contains electrical components.

5. Service & maintenance records

Order	Serial	Date
--------------	---------------	-------------

Date	Date
Sign	Sign
Remarks	Remarks

Date	Date
Sign	Sign
Remarks	Remarks