




PRESSALIT®

<b>Product</b> <b>Item number</b>	<p>MATRIX powered basin unit</p> <p>R4875000</p>
<b>Colour</b>	<p>EAN:5708590349749</p> <p>White</p> 
<b>Product description</b>	<p>MATRIX brackets are tailor made for MATRIX wash basins. They are available in electrically and manually height adjustable versions, as well as fixed versions, all of them with a hygienic cover to conceal the plumbing installations.</p> <p>MATRIX wash basin bracket, wall-mounted, with white installation cover and incl. concealed feed and waste system. Electrically height adjustable 200 mm with operating buttons in front edge of wash basin. Incl. fixing bolts for wash basin (Can only be used with MATRIX SQUARE wash basins, R2170/R2172). Materials: Powder-coated steel and polystyren Excl. mounting kit.</p>
<b>Adjustability</b> <b>Adjustment height</b>	<p>Height adjustable</p> <p>200 mm</p>
<b>Product capacity</b>	<p>Maximum load: 150 kg. Suitable for wash basins that weigh 30/40 kg. When the wash basin is loaded with 150 kg, each wall fastener will be carrying 60 kg. <i>Wash basin brackets need to be able to carry the weight of the wash basin, as well as support the weight of the person who is leaning against the wash basin or who is using the wash basin to rise from seated position.</i> <b><i>Neither wash basin nor wash basin brackets are intended to carry a person, and the test methods applicable to wash basin brackets reflect this: ISO 17966 prescribes a static test on the front edge of the wash basin at 75% of maximum user weight, i.e. when the maximum load on a wash basin bracket is given as 150 kg and the wash basin weighs 40 kg, it is – according to ISO 17966 – the equivalent of a user weighing 146 kg leaning on the front edge of the wash basin.</i></b></p>
<b>Speed</b>	<p>Speed up mm: 11 mm/s Speed down mm: 5 mm/s</p>

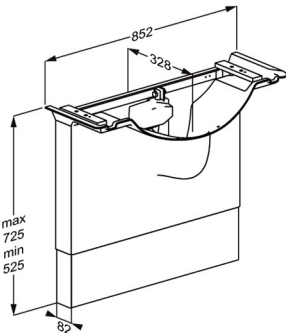

<b>Product</b>	MATRIX powered basin unit
<b>Item number</b>	R4875000
<b>Noise level</b>	48 dB
<b>Materials</b>	Stainless steel and ABS with acrylic.
<b>Mains connection (V)</b>	230
<b>Actuator (motor)</b>	LA31 Careline (Linak), 1500N, IPX6
<b>Control box</b>	CB9 Careline (Linak), IPX6
<b>Power consumption (maximum load)</b>	Power consumption by maximum load, watt: 70 W Power consumption by standby: 1,5 W
<b>Warranty</b>	Pressalit makes a commitment to remedy material and manufacturing defects for a period of five years from the invoicing date.
<b>Cleaning</b>	<p>VALUE &amp; MATRIX wash basin brackets.</p> <p>The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use corrosive or abrasive cleaners as it may result in the surface being damaged.</p> <p>For surface disinfection, chlorine-based disinfectants may be used according to the product specification and safety precautions for these products.</p> <p>The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.</p> <p><b>DISINFECTANTS:</b> Clinimax Difficil-S Erisan OXY+ Klorilli</p> <p><b>NON-DISINFECTANTS:</b> Suma Star free D1 Taski 100 Taski 200</p> <p>The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.</p>
<b>WEEE</b>	Please enquire of your local Pressalit subsidiary or distributor (in countries where there is no subsidiary) that will arrange for recycling of electronic waste in accordance with the WEEE directive.



# PRESSALIT®


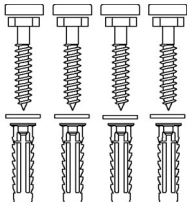

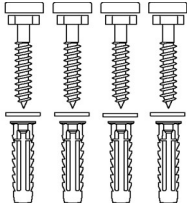
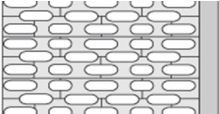
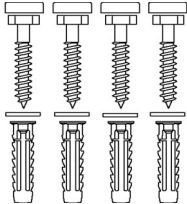

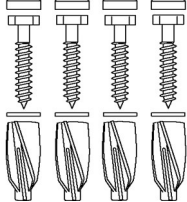
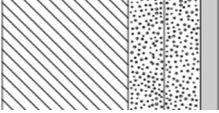
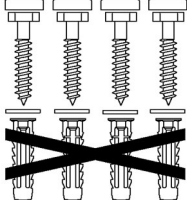
<b>Product</b>	MATRIX powered basin unit
<b>Item number</b>	R4875000
<b>Declaration of conformity</b>	This product is in compliance with all relevant requirements of regulation (EU) 2017/745 on medical devices. It also complies with all relevant requirements in the Council Directive 2006/42/EC (of 17 May 2006) on machinery and the Council Directive 2011/65/EC on RoHS.










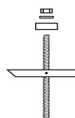






<b>Product</b>	MATRIX powered basin unit
<b>Item number</b>	R4875000

## Installation in different wall types

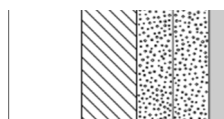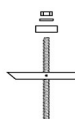	<p><b>Concrete &gt;B25</b></p> <p>V8605 - Mounting kit, 4 coach screws Ø10 x 70, plugs SX12, with grey screw covers. To be ordered separately.</p>	
	<p><b>Solid Brick &gt;= Mz12</b></p> <p>V8605 - Mounting kit, 4 coach screws Ø10 x 70, plugs SX12, with grey screw covers. To be ordered separately.</p>	
	<p><b>Solid Sand Lime Brick &gt;= Hlz12</b></p> <p>V8605 - Mounting kit, 4 coach screws Ø10 x 70, plugs SX12, with grey screw covers. To be ordered separately.</p>	
	<p><b>Aerated Concrete P4</b></p> <p>V8655 - Mounting kit, 4 coach screws Ø10 x 70, plugs GB14, with grey screw covers. To be ordered separately.</p>	
	<p><b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood</b></p> <p>V8605 - Mounting kit, 4 coach screws Ø10 x 70, plugs SX12, with white screw covers. NB! Plugs are not needed. To be ordered separately.</p>	

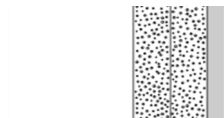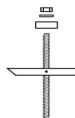

# PRESSALIT®

<div>Product</div> <div>Item number</div>	<div>MATRIX powered basin unit</div> <div>R4875000</div>			
	<div><b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood</b></div> <div>Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.</div>			
	<div><b>Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (2 mm)</b></div> <div>Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.</div>			
	<div><b>Drywall (2 x 12,5 mm) water-resistant</b></div> <div>Wall fasteners (4 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.</div>			

It is the responsibility of the installer to ensure that suitable wall fasteners are used.



[www.pressalit.com](http://www.pressalit.com)

Pressalit A/S  
 Pressalitvej 1  
 DK-8680 Ry  
 Tel.: +45 8788 8788  
 Fax: +45 8788 8789  
 E-mail: [pressalit@pressalit.com](mailto:pressalit@pressalit.com)

# PRESSALIT®