

SP50 and SP80 Installation Instructions when used without Throne Rails


1 x Throne Spacer (SP50 or SP80) 1 x Rear Spacer Plate (not required if using Throne Rails) 2 x M10 Bolts 2 x Steel Washers 2 x Nylon Washers 2 x Nuts


Installation

- Spacer is to fit between Toilet Bowl and Toilet Seat and Lid
- 2. Remove the existing Toilet Seat and Lid
- 3. Fit Spacer onto Toilet Bowl along with Spacer Plate
- 4. Refit Toilet Seat and Lid and replace the existing bolts with the new longer bolts supplied with the Spacer
- 5. Tighten Bolts sufficiently so that the Spacer and Seat are firm on the Toilet Bowl. Do not overtighten!

Care and Maintenance

1. The Spacer can be cleaned with a solvent free detergent with a PH of 6-9, or 70% disinfectant solution.

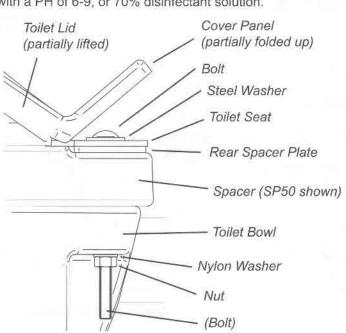

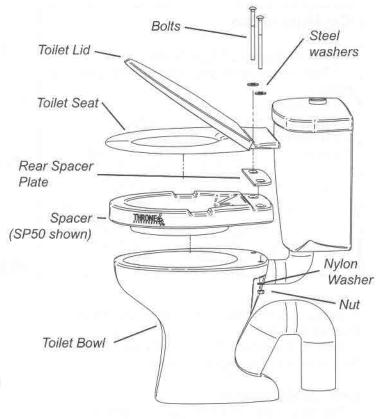

Exploded View



Assembled on Toilet