



The **Relaxa-Back** will give support when getting on to the belt and lowering into the bath. When the Relaxa-Back is moved to the back of the bath, it can be used as a head rest.

The **Relaxa-Seat** makes getting on to the belt simple and safe. The padded seat can be raised or lowered and is an ideal place to sit in comfort when drying.



## Technical Specifications

Thermometer	The unit has a thermometer to test the temperature of the water.
Power	Two 12 volt rechargeable lead acid batteries provide a 24 volt DC supply to the motor. Indicator lights in green and red shows the battery is in operation and the battery condition. A red light will appear to indicate that the battery is low together with an audible alarm. An empty battery symbol will appear when the battery is too low. The bath-lift will not work in the down position but will bring you back out of the bath.
Battery Charger	A managed 24 volt 300MA charger is provided.
Maximum Load	127 kg (20 stone)
Controls	"UP" and "DOWN" buttons are located on the front of the bath-lift. A sealed hand control with "UP" and "DOWN" buttons is also provided.
Belt Reversal	The Relaxa is designed with an anti-belt reversal mechanism.
Handle	An integral handle is provided but this can be removed or re-sited as required.
Belt	Reinforced Polypropylene
Dimensions	The unit is slim and fits neatly at the side of the bath. The length is 55.5cm, the height is 32.5cm and the depth is 11.8cm
Weight	The unit itself weighs 11kgs. The box when packed to include other items weighs 15.9kgs
Manufactured	The complete unit is manufactured in the United Kingdom

**relaxa**  
mobility

### For further information please contact

1 Kites Park, Summerleys Road,  
Princes Risborough, Buckinghamshire, HP27 9PX  
t: 01844 397044

[www.relaxamobility.co.uk](http://www.relaxamobility.co.uk)

Registered in England and Wales: 12076361

Relax with a  
**relaxa**  
mobility

Bathlift



- Easy to fit
- Battery powered NOT mains electricity
- Audible alarm when battery is low
- Remote control as standard
- Thermometer for water temperature
- Does not obstruct the bath
- Provides independence for those that find the bath difficult to enter or leave.



# How does it work

**1** Simply push in the re-chargeable battery.



**4** Sit on the belt and by touching a button, be gently lowered into the bath.



Feed out the belt and slip on belt sleeve. **2**



Relax and enjoy the luxury of a good soak. **5**

**3** Locate the belt on the floor mounting bar and tension.



**6** Touch a button and be gently raised out of the bath.



Audible alarm and graphics when battery is low. In addition there is a remote hand control and a temperature probe to register the water temperature and avoid scalding.

