

EN Troubleshooting Guide Height Adjustable Nursing tables 343, 345, 346

Products

Changing table 343
Changing table 343 - bathtub
Nursing bench 345
Shower trolley 346

The nursing table / shower trolley is designed and tested for optimal reliability and service life, provided that it is operated, serviced and inspected in accordance with these instructions. Should any problem arise, consult the troubleshooting list below for guidance.

Unload the work surface prior to any troubleshooting or repairs. Troubleshooting, service and repairs must be carried out by competent persons.

If the problem persists after taking the steps listed below, contact a qualified repair person or your supplier

| Symtom | Action |
|----------------------------------------------------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| The table does not move when the push-button is pressed | Check that the fuse for the mains feed is not damaged. Check that the mains plug is connected. |
| The motors are not moving parallel | Lower the table so one if the motors are in its lowest position. Press the down button for 5 seconds until the control box clicks. When the control box clicks the synchronisation is ok. |