



EXCEL RANGE OF ASSISTED BATHS

USER GUIDE

EXCEL 300 – Fixed Bath with Powered Swing Seat

To lift the seat out of the bath, press the “Seat out” button. This will lift the seat up. Continue to press the “Seat out” button and the seat will automatically swing out of the bath and then lower on the outside. Continue to press until the seat is at the required height.

To put the seat into the bath, press the “Seat in” button to raise it up. Continue to press the button and the seat will automatically swing over the bath and then lower into the bath.

N.B. You can stop the seat at any point by lifting your finger off the button, should you wish to assist the bather in any way.

Ensure there is no obstruction in the way of the seat as it swings out of the bath.

EXCEL 400 – Height-adjustable bath only

To lift the bath, press the “Bath up” button until the bath reaches the required height. To lower the bath, press the “Bath down” button.

EXCEL 600 – Height-adjustable bath with Powered Swing Seat

To lift the seat out of the bath, press the “Seat out” button. This will lift the seat up. Continue to press the “Seat out” button and the seat will automatically swing out of the bath and then lower on the outside. Continue to press until the seat is at the required height.

To put the seat into the bath, press the “Seat in” button to raise it up. Continue to press the button and the seat will automatically swing over the bath and then lower into the bath.

N.B. You can stop the seat at any point by lifting your finger off the button, should you wish to assist the bather in any way.

Ensure there is no obstruction in the way of the seat as it swings out of the bath

To lift the bath press the “Bath up” button until the bath reaches the required height. To lower the bath, press the “Bath down” button.

N.B. The bath will only rise or lower when the seat is fully lowered inside it.

The seat will only rise or lower when the bath is in the fully lowered position.

GENERAL POINTS;

N.B. DO NOT turn the power off to the bath as the batteries will lose their charge and the seat/bath will then not move. If the power is off for any length of time, please allow several hours for the batteries to re-charge before attempting to use the bath. However, if the power is off for a prolonged period, the batteries may discharge their power to a point where they cannot be re-charged and they will have to be replaced. If it is known that the mains power is going to be lost for a long period, unplug the battery cable from the control box to avoid loss of charge to the batteries.

Cleaning

The bath may be cleaned using most mild cleaning agents.

Chemical agents and solvents must not be used, i.e. drain/oven cleaners, acetone, or acids.

The taps, side and end panels and handset should be cleaned using a warm soapy solution and wiped dry.

Guarantee

This product is guaranteed against all manufacturing defects for one year. This does not affect your statutory rights.