PRESSALIT®

R8402  
R8403

## SCT 1000

**Shower change table – Operation and maintenance manual** en

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# Content

Mounting instruction .....	3
Operational signage .....	3
Symbols used in this manual .....	4
Specific safety .....	4
General safety .....	5
Product label .....	6
Intended use .....	8
Intended operator profile .....	8
Intended user profile .....	8
Operating environment .....	8
Versions .....	9
Operating instructions .....	10
Height adjustment .....	11
Cleaning .....	12
Disinfection .....	12
Maintenance .....	13
Maintenance inspection .....	13
Trouble-shooting .....	15
Technical data .....	16
Dimensions (mm) .....	17
Dimensions (inches) .....	18
Disposal and recycling .....	19

## Mounting instruction

en

A mounting instruction is included with the product. It can also be downloaded from [www.pressalit.com](http://www.pressalit.com).

## Operational signage

An operational signage is included with the product. It can also be downloaded from [www.pressalit.com](http://www.pressalit.com).

# Symbols used in this manual

The following symbols are used in this manual as well as the associated documentation supplied with the product.

**WARNING** indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

**CAUTION** indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices.

**IMPORTANT** indicates important information about handling and use of the product.

**NOTICE** is used to address practices not related to personal safety.

Hand injury warning

Electricity or potentially dangerous voltage.

Head injury warning.

## Specific safety

- Never leave a person lying on the shower table without supervision.
- Always be careful when moving a person to and from the shower table.

# General safety

## **WARNING**

- Read and understand the manual completely before use. Keep the manual close to the unit.
- Comply with this manual to avoid accidents involving serious personal injury.
- Never use or handle this product in other ways than are specified in this manual, as this can put your safety at risk and cause damage to the product.
- Make sure that everyone who is to install or use the product and its systems has the necessary information and access to this manual.

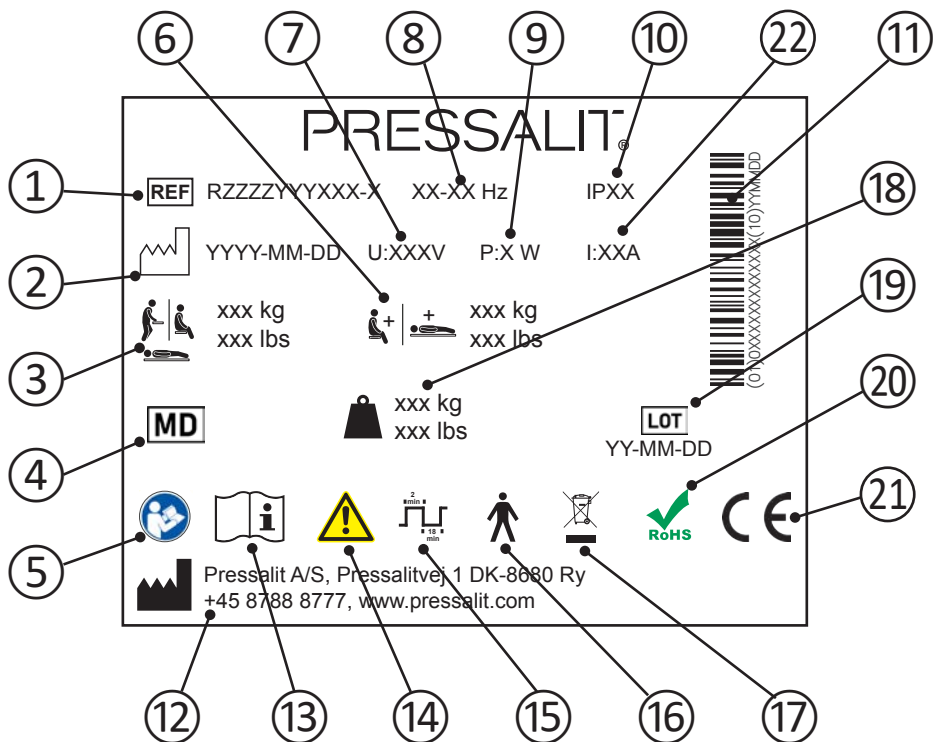
## **CAUTION**

- Please comply with these instructions to avoid the product becoming damaged or destroyed.
- Do not perform repairs, disassembly, or assembly operations, extensions, re-adjustments or modifications to this product. These must be carried out by Pressalit or by persons authorized by Pressalit only.
- Never allow children to operate this product unless under supervision. The product is not intended for play.
- Never use the product if it is defective or damaged. Do not use a defective product before it has been repaired.

## **NOTICE**

- The information in this manual is based on correct fitting of the product in accordance with our Assembly instruction.
- Pressalit cannot be held liable if the product is used in any way which differs from that described in the guide.
- We reserve the right to amend this manual without prior notice.

# Product label

en

① Article number

② Production date

③ Maximum user weight

④ Medical device

⑤ Consult manual for use

⑥ Max load

⑦ Mains voltage

⑧ Frequency

⑨ Maximum power consumption

⑩ IP class

---

⑪ GS1-128 barcode

---

⑫ Name and address of manufacturer

---

⑬ Consult manual for use

---

⑭ Consult the manual for important cautionary information such as warnings and precautions

---

⑮ Duty cycle

---

⑯ Type B applied part Part of the product in normal use necessarily comes into physical contact with the patient to perform its function  
The product complies with requirements of IEC 60601-1 to provide protection against electric shock

---

⑰ Do not dispose as unsorted municipal waste. The product should be collected separately and returned to the designated recycling service.

---

⑱ Product weight

---

⑲ Batch code

---

⑳ This product does not contain any of the hazardous substances mentioned in the Council Directive 2011/65/EC on RoHS.

---

These products are CE-marked in accordance with relevant requirements in:

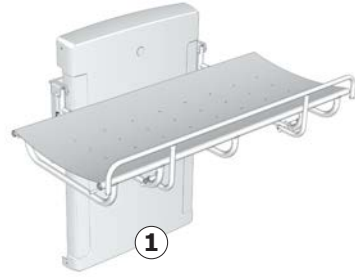
㉑ - Regulation (EU) 2017/745 (5 April 2017) on medical devices.  
- Council Directive 2006/42/EC (of 17 May 2006) on machinery.  
- Council Directive 2011/65/EC (of 8 June 2011) on RoHS.  
A declaration of conformity can be found at [www.pressalit.com](http://www.pressalit.com).

---

㉒ Electric Current

---

Product label placement ①.

## Intended use

This product is intended for bathing a person who is unable to stand up or sit down. The person is lying horizontally on the shower table.

## Intended operator profile

The product is to be used by personnel trained in health care and trained in washing a person in their care.

## Intended user profile

The product can be used by all - adults, children, young, old, whose weight does not exceed the **maximum load of 150 kg / 330 lbs.**

## Operating environment

For indoor use in a wet environment, such as a bathroom or wash room.  
The product is intended to be used at room temperature, in private homes, nursing homes and hospitals under normal working conditions.

### **CAUTION**

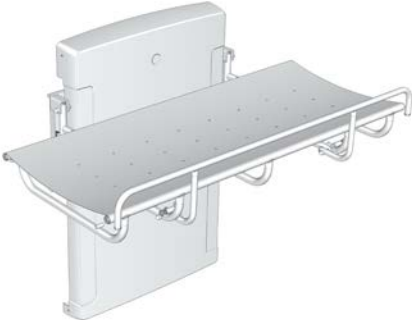
The surface of the product may become hot and / or cold if exposed to external sources of heat or cold (eg sunlight or frost).

### **NOTICE**


Any serious incident that has occurred in relation to the product should be reported to the manufacturer and the health authorities.

# Versions


R8402

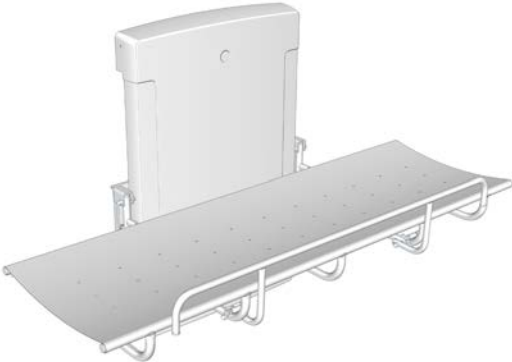R8403

en

# Operating instructions

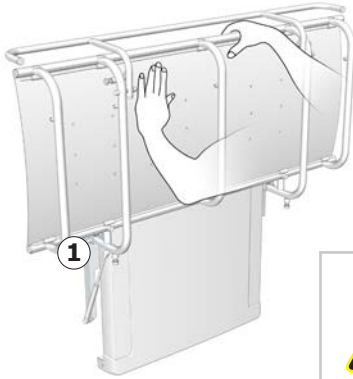
Support the shower table using your hand or body when folding it down, to avoid the risk of damage.

The built-in gas cylinders make it easy to fold the shower table up against the wall after use.


**Take care to avoid injury to hands when in the vicinity of closing mechanical parts ①.**

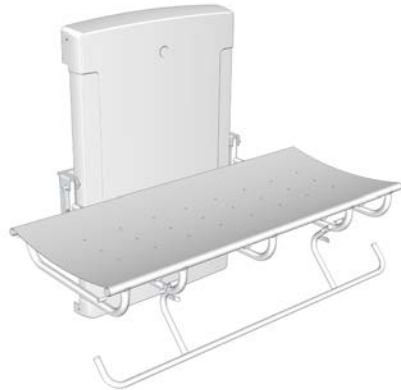**To pull the safety rail up:** Swing the rail upwards from under the shower table. The safety rail will automatically lock into position.


en

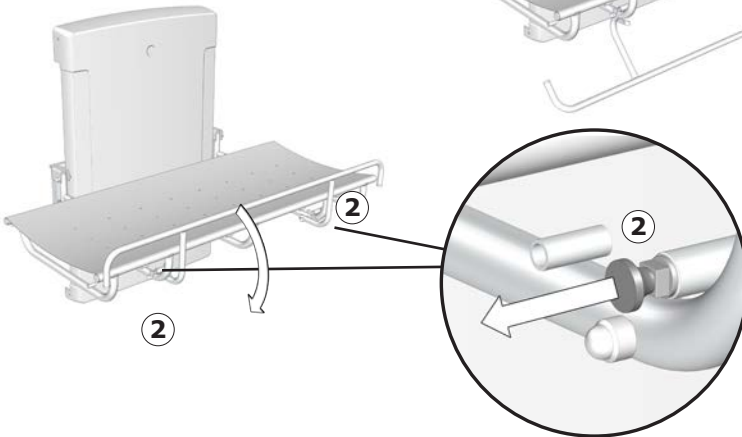**To fold the safety rail away:** Release the two locks ② simultaneously and swing the safety rail down under the shower table.

# Height adjustment

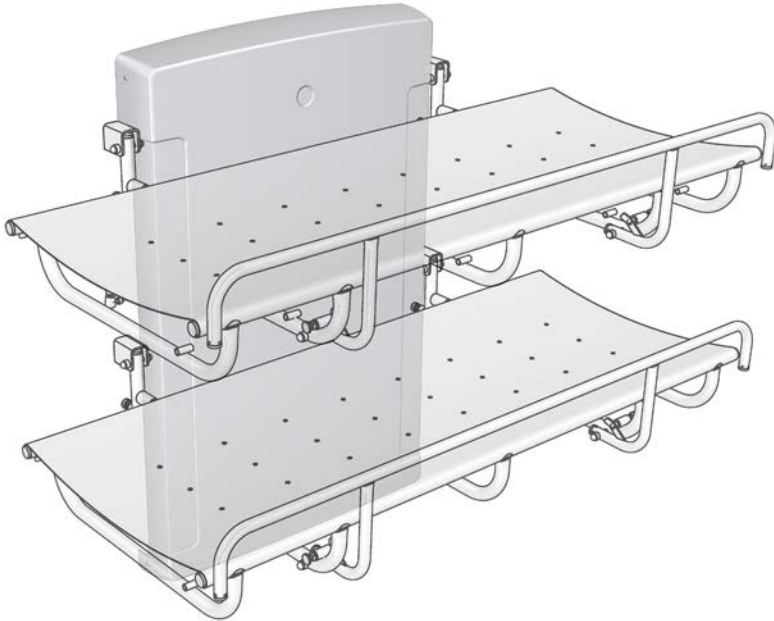When the shower table is installed at the recommended height, it can be adjusted between 480 and 880 mm ( 18.9" and 34.6" ) above the floor.

The height adjustment is controlled by the wired remote control unit.

Press and hold the Up button to raise the shower table, and press and hold the Down button to lower the shower table.

**Make sure that there is no objects under the shower table when operating the height adjustment system.**

# Cleaning

Clean the shower table after each use. Use ordinary soapy water. Rinse off with fresh water and dry with a soft cloth.

## IMPORTANT

- Do not use an autoclave or other type of steam cleaner.
- Avoid aggressive cleaning agents containing acids or chloride.
- Never spray the shower table directly with a high pressure cleaner.
- Avoid use of aggressive, abrasive, corrosive or chloride-based cleaning or disinfecting products.

## Disinfection

When using the shower table for multiple patients, it is necessary to both clean and disinfect the shower table between uses on different patients to prevent the risk of cross infection.

A list of suitable cleaning agents and disinfectants can be found at [www.pressalit.com](http://www.pressalit.com)

## CAUTION

Always read the material safety data sheet (MSDS) and supplied instructions of the cleaning, disinfecting or de-scaling agent and comply with instructions for safe use, dilution, rinsing, drying, handling and storage of the agent.

## NOTICE

If the product is installed in an atmosphere heavy in salt or chloride, the warranty will be reduced from the normal 3 years to 1 year.

Salt and chloride have a corrosive effect on the products components and will cause a decrease in the life expectancy of the product. The damaging effects can, however, be reduced if the product is showered with tap water on a daily basis.

# Maintenance

Expected service life of the shower table is 10 years, provided that maintenance is carried out as described below:

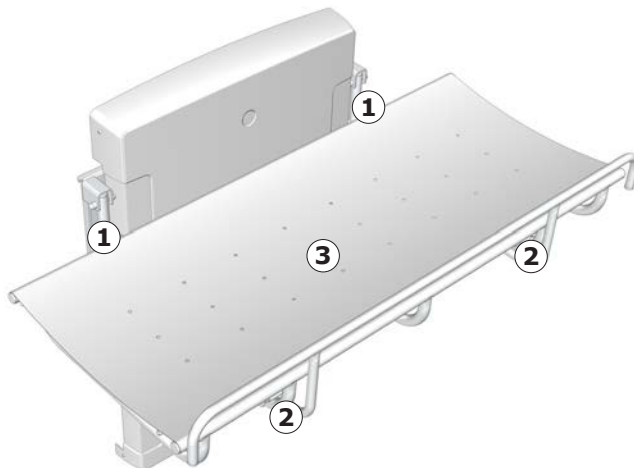
It is recommended on a weekly basis to run the motors on the product to their maximum positions by raising the shower table to its full height, and then lowering it to its lowest position, so that the internal parts are lubricated.

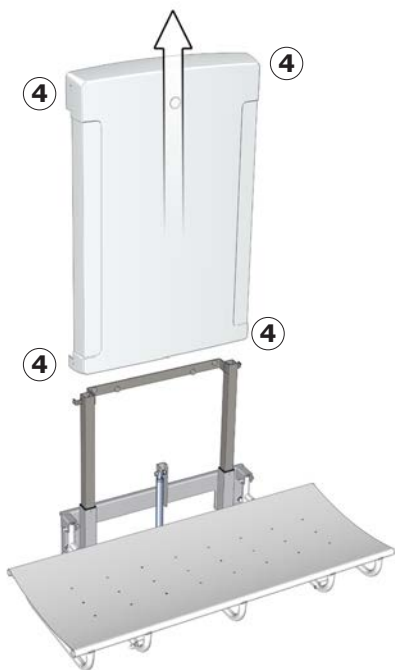
It is recommended that maintenance inspection is done once a year.


## Maintenance inspection

- Check the hinges ① on the shower table and the safety rail ② . Tighten if necessary.
- Check the canvas ③ for damage and replace if necessary.

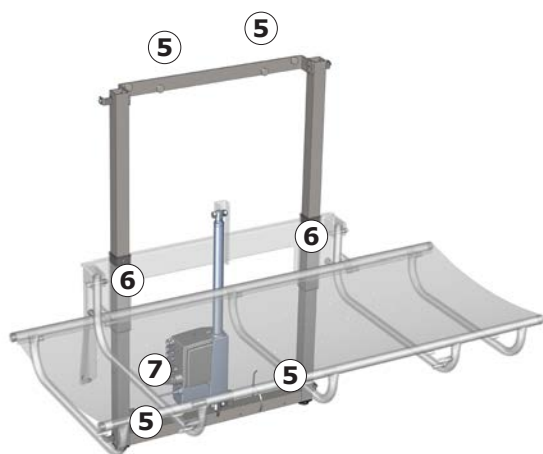- Remove the upper and lower retaining screws ④ on the cover and remove it carefully.

- Check the wall brackets ⑤ and tighten if necessary.
- Oil moving parts, especially the sliding brackets ⑥.
- Check all electrical connections to the control box ⑦.
- Put the unit cover back again and tighten the retaining screws.
- Reconnect mains.

# Trouble-shooting

Symptom	Cause	Action
The shower table does not stay in upright position.	The gas cylinders is defective.	Contact Pressalit or your local dealer.
The product will not work, even though the remote control is used correctly.	Fault in power supply.	Check that all wires and plugs are intact and correctly connected. Contact Pressalit or your local dealer.
The shower table runs up and down unevenly.	The sliding brackets are not properly lubricated.	Oil the sliding brackets as described under "Maintenance".

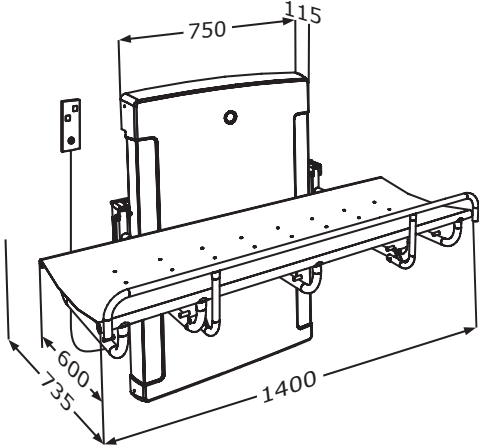
Contact Pressalit or your dealer if the problem cannot be remedied. Do not attempt to repair the product yourself, as this will invalidate the guarantee.

# Technical data


Max load	150 kg /330 lbs
Materials	Powder laquered stainless steel Plastics: PVC (canvas) Polystyrene (cover)
Weight	R8402: 44.46 kg, 98.01 lbs R8403: 45.92 kg, 101.23 lbs
IP class	The product has an ingress protection class rating of IPX6. The product may not be immersed in water, but is designed to withstand powerful jets (12.5 mm nozzle) of water from any direction.
<b>Power specifications</b>	
Max consumption	70 W
Voltage	Mains connection: 230V/120V/100V
Frequency	50/60 Hz
Current	0,3 A

# Dimensions (mm)


**R8402**

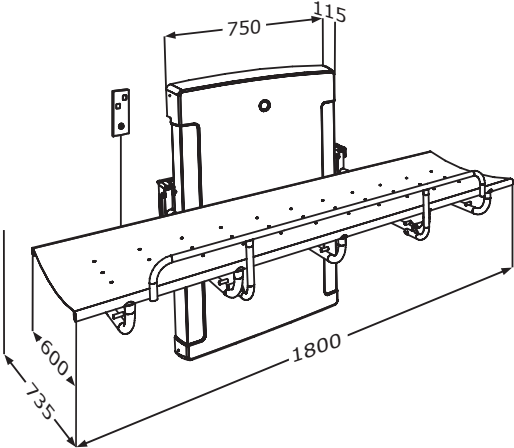
**R8403**

en
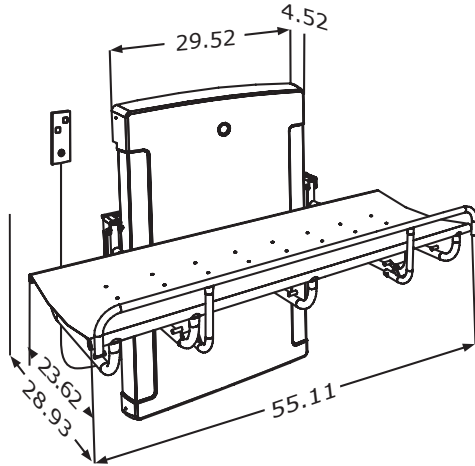
# Dimensions (inches)

**R8402**

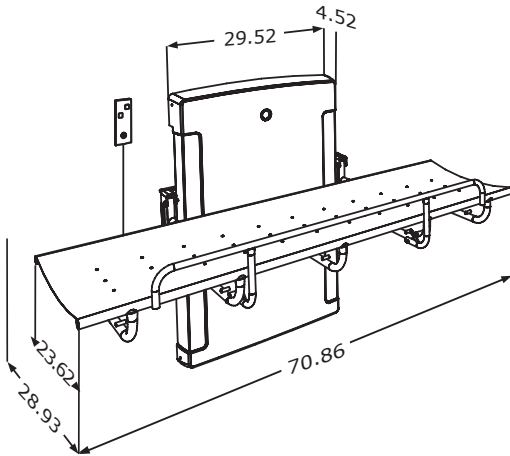**R8403**

en

# Disposal and recycling

The unit contains reusable materials. There are no known hazards or risks associated with disposal of the product. All components can be discarded after having been cleaned and disinfected.

When disposing of the product, we recommend that it be disassembled and broken down into different waste groups for recycling or combustion.

The following table provides information on the recycling and handling of the product.

<b>Recyclable components</b>	<b>Material</b>
Steel	Steel can be recycled by being re-melted and included as a secondary material in the production of new steel.
Plastics	Use plastic sorting and recycling systems where they are locally available, otherwise incinerating plastic has the added benefit of generating energy which can be used for, for example, combined heat and power production.
Electronics	The electronic components should be collected separately and returned to the designated recycling service.

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