PRESSALIT®

R2120 - R2121
R2140 - R2141
R2142 - R2143

R4820 - R4825
R4826
R4840 - R4842
R4845 - R4847

MATRIX wash basin lifter – Operation and maintenance manual	en
MATRIX vaskeophæng – Brugs- og vedligeholdelsesvejledning	dk
MATRIX Waschtisch-Lifter – Gebrauchs- und Pflegeanleitung	de
MATRIX support de lavabo – Manuel d'utilisation et d'entretien	fr
MATRIX muurframe voor wastafel – Gebruiks- en onderhoudshandleiding	nl
MATRIX upphängning till tvättställ – Bruks- och underhållsanvisning	sv
MATRIX oppheng til servant – Bruks- og vedlikeholdsveiledning	no
MATRIX soporte de lavabo - Instrucciones de uso y mantenimiento	es
MATRIX wspornik umywalki – Instrukcja obsługi i konserwacji	pl
MATRIX 洗面盆支架 - 操作和维护手册	中文

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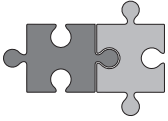
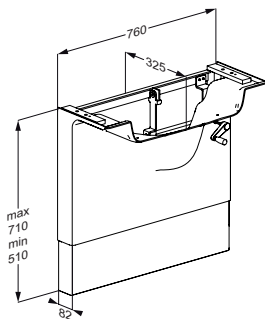
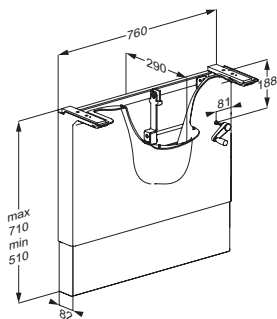
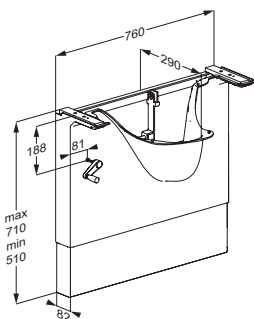
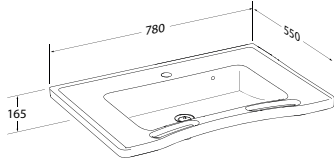
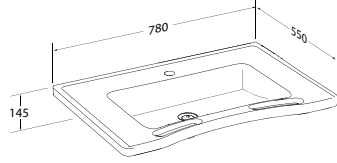
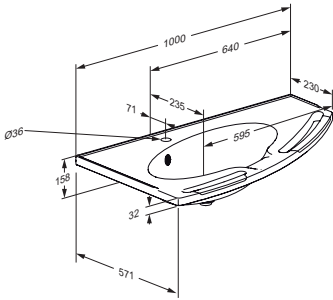
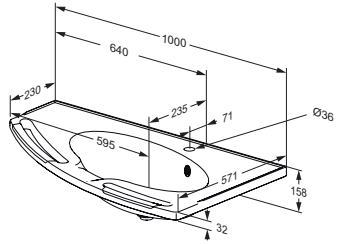
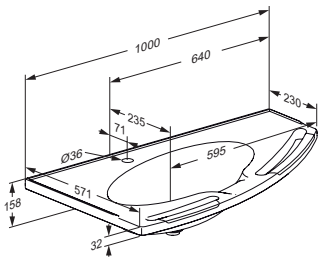
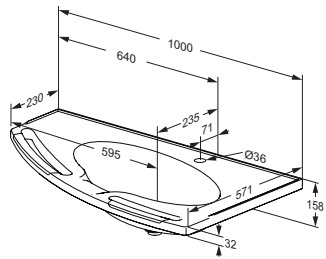
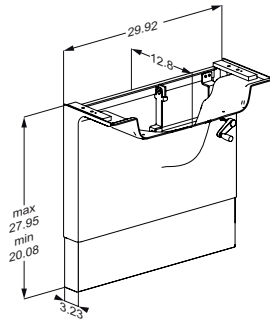
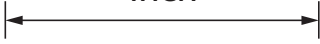
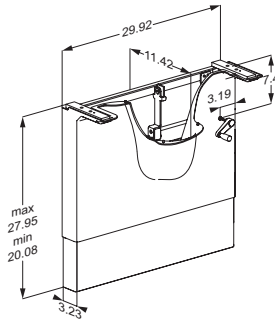
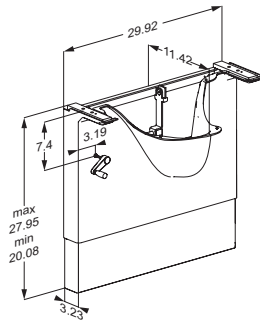
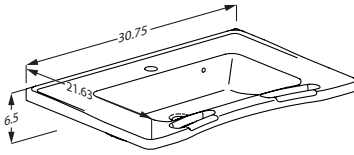
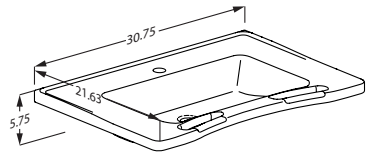
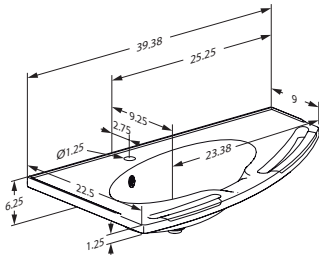
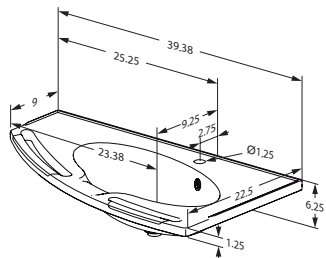
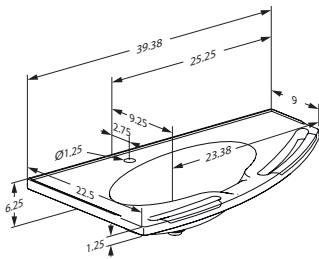
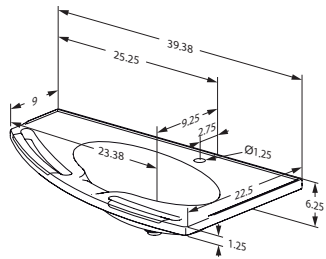
R2143

Content

Mounting instruction	9
Symbols used in this manual	10
General safety	11
Product label	12
Intended use	13
Intended operator profile	13
Operating environment	13
Operating instructions	14
Cleaning	15
Maintenance	16
Maintenance inspection	16
Trouble-shooting	17
Technical data	17
Disposal and recycling	17

Mounting instruction

A mounting instruction is included with the product. It can also be downloaded from www.pressalit.com.

Symbols used in this manual

The following symbols are used in this manual as well as the associated documentation supplied with the product.

WARNING indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

CAUTION indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices.

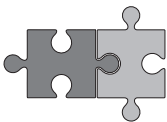
IMPORTANT indicates important information about handling and use of the product.

NOTICE is used to address practices not related to personal safety.

Hand injury warning

Compatible products

General safety

WARNING

- Read and understand the manual completely before use. Keep the manual close to the unit.
- Comply with this manual to avoid accidents involving serious personal injury.
- Never use or handle this product in other ways than are specified in this manual, as this can put your safety at risk and cause damage to the product.
- Make sure that everyone who is to install or use the product and its systems has the necessary information and access to this manual.

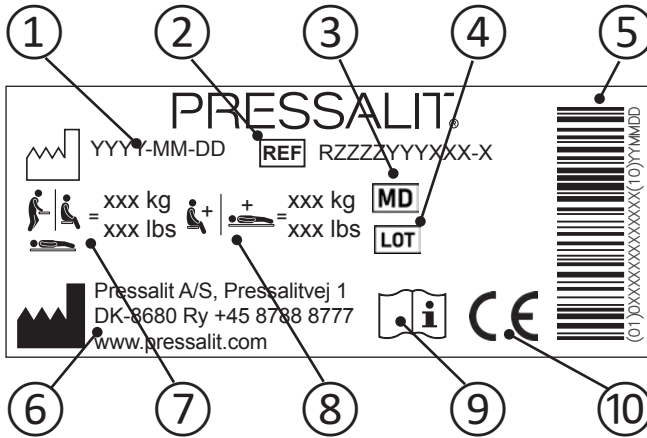
CAUTION

- Please comply with these instructions to avoid the product becoming damaged or destroyed.
- Do not perform repairs, disassembly, or assembly operations, extensions, re-adjustments or modifications to this product. These must be carried out by Pressalit or by persons authorized by Pressalit only.
- Never allow children to operate this product unless under supervision. The product is not intended for play.
- Never use the product if it is defective or damaged. Do not use a defective product before it has been repaired.

NOTICE

- The information in this manual is based on correct fitting of the product in accordance with our Assembly instruction.
- Pressalit cannot be held liable if the product is used in any way which differs from that described in the guide.
- We reserve the right to amend this manual without prior notice.

Product label

① Production date

② Article number

③ Medical device

④ Batch code

⑤ GS1-128 barcode

⑥ Name and address of manufacturer

⑦ Maximum user weight

⑧ Max load

⑨ Consult manual for use

⑩ These products are CE-marked in accordance with relevant requirements in:

- Regulation (EU) 2017/745 (5 April 2017) on medical devices.
 - Council Directive 2006/42/EC (of 17 May 2006) on machinery.
- A declaration of conformity can be found at www.pressalit.com.

Product label placement ①.

Intended use

This product is intended for optimum ergonomics at a wash basin, for both seated and standing users with disabilities.

Intended operator profile

The product can be used by all - adults, children, young and old.

Operating environment

For indoor use in a wet environment, such as a bathroom or wash room, in private homes, nursing homes and hospitals.

en

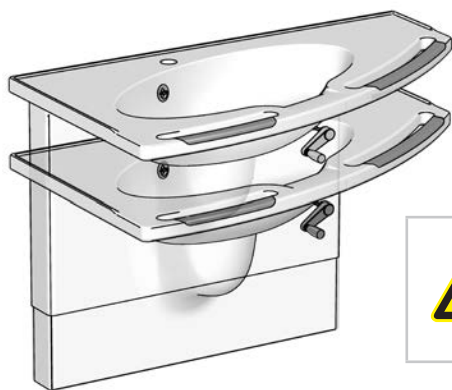
CAUTION

The surface of the product may become hot and / or cold if exposed to external sources of heat or cold (eg sunlight or frost).

NOTICE

Any serious incident that has occurred in relation to the product should be reported to the manufacturer and the health authorities.

Operating instructions

The wash basin lifter can be adjusted 200 mm up and down.
Max load is 150 kg.

Take care to avoid injury to hands when in the vicinity of moving mechanical parts.

Raise

Lower

Turn the handle clockwise to raise the wash basin, and counterclockwise to lower it.

CAUTION

Never place objects underneath the wash basin.

Cleaning

The product is to be cleaned with hot water or a mild soapy cleaning agent. A list of suitable cleaning agents/disinfectants tested by Pressalit can be found on www.pressalit.com.

CAUTION

Always read the material safety data sheet (MSDS) and supplied instructions of the cleaning, disinfecting or de-scaling agent and comply with instructions for safe use, dilution, rinsing, drying, handling and storage of the agent.

IMPORTANT

- Do not use an autoclave or other type of steam cleaner.
- Avoid aggressive cleaning agents containing acids
- Never spray the product directly with a high pressure cleaner.
- For surface disinfection, chlorine-based disinfectants may be used according to the product specification and safety precautions for these products.

en

Maintenance

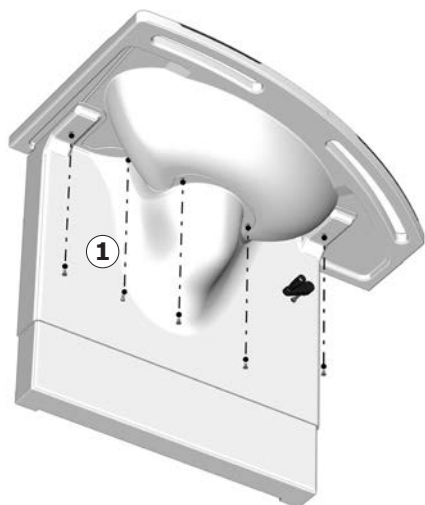
Expected service life of the wash basin lifter is 10 years, provided that maintenance is carried out as described below.

It is recommended on a monthly basis to adjust the product to its maximum positions up and down to lubricate the internal parts.

Maintenance inspection

Take the wash basin to its highest position.


Remove the screws and the upper front cover ①.

Lubricate the inner square tubes ② and the hoisting mechanism ③.

Take the wash basin to its lowest position, and then run it to the top position again to distribute the lubricant.

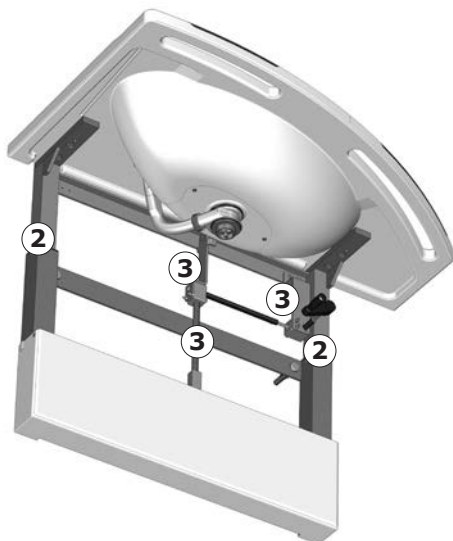Reattach the front covers.

en

Trouble-shooting

Symptom	Cause	Action
The product runs up and down unevenly.	The sliding brackets are not properly lubricated.	Oil the sliding brackets as described under "Maintenance".

Contact Pressalit or your dealer if the problem cannot be remedied. Do not attempt to repair the product yourself, as this will invalidate the guarantee.

Technical data

Max load	150 kg / 330 lbs
Materials	Stainless steel Plastics
Weight	R4820/25/26: 19 kg / 42 lbs R4840/42/45/47: 15 kg / 33 lbs

Disposal and recycling

The unit contains reusable materials. There are no known hazards or risks associated with disposal of the product. All components can be discarded after having been cleaned and disinfected.

When disposing of the product, we recommend that it be disassembled and broken down into different waste groups for recycling or combustion.

The following table provides information on the recycling and handling of the product.

Recyclable components	Material
Steel	Steel can be recycled by being melted down and included as a secondary material in the production of new steel.
Plastics	Use plastic sorting and recycling systems where they are locally available, otherwise incinerating plastic has the added benefit of generating energy which can be used for, for example, combined heat and power production.

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