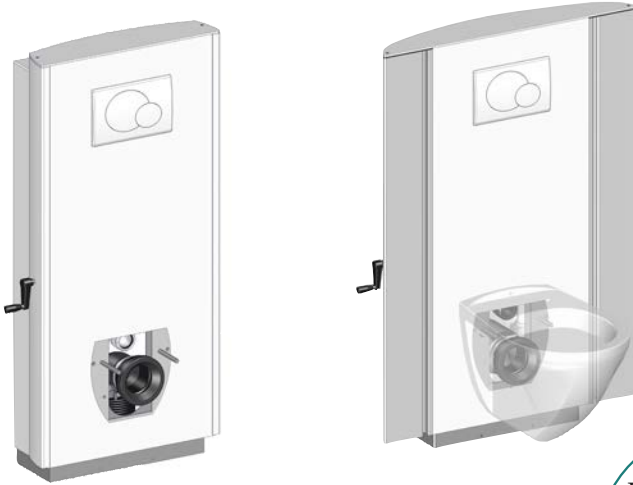
PRESSALIT®

R8012, R8022
R8032, R8042
R8112, R8122
R8132, R8142

SELECT TL1 & TL2 toilet lifter – Operation and maintenance manual	en
SELECT TL1 & TL2 toiletløfter – Brugs- og vedligeholdelsesvejledning	dk
SELECT TL1 & TL2 WC-Lifter – Gebrauchs- und Pflegeanleitung	de
SELECT TL1 & TL2 WC réglable en hauteur – Manuel d'utilisation et d'entretien	fr
SELECT TL1 & TL2 hoog-laag-toiletsysteem – Gebruiks- en onderhoudshandleiding	nl
SELECT TL1 & TL2 Upphängning till toalett – Bruks- och underhållsanvisning	sv
SELECT TL1 & TL2 Oppheng til toalett – Bruks- og vedlikeholdsveiledning	no
SELECT TL1 & TL2 Soporte de inodoro - Instrucciones de uso y mantenimiento	es
SELECT TL1 i TL2 Podnośnik toalet – Instrukcja obsługi i konserwacji	pl
SELECT TL1 & TL2 坐便器升降装置 — 操作和维护手册	中文
SELECT TL1 & TL2 رافعة كرسي المراض – دليل التشغيل والصيانة	العربية

English	8
Dansk	19
Deutsch	30
Français	41
Nederlands	52
Svenska	63
Norsk	74
Español	85
Język polski	96
中文	107
ةيبرعلا	118

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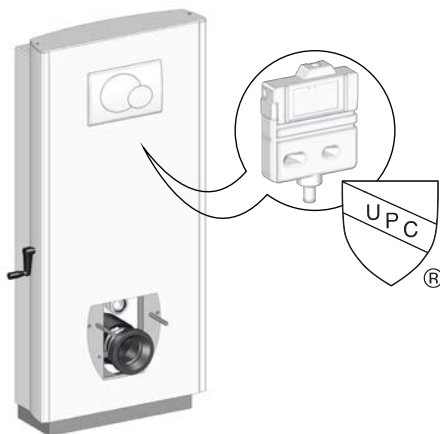
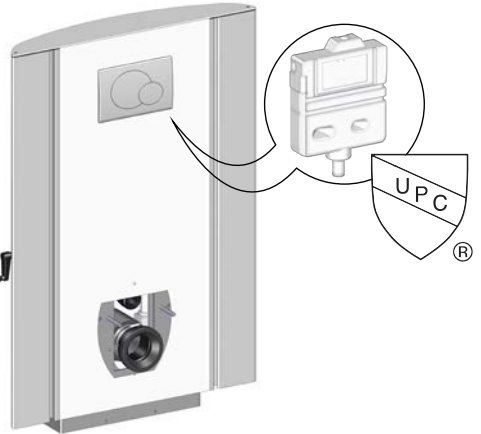
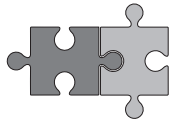
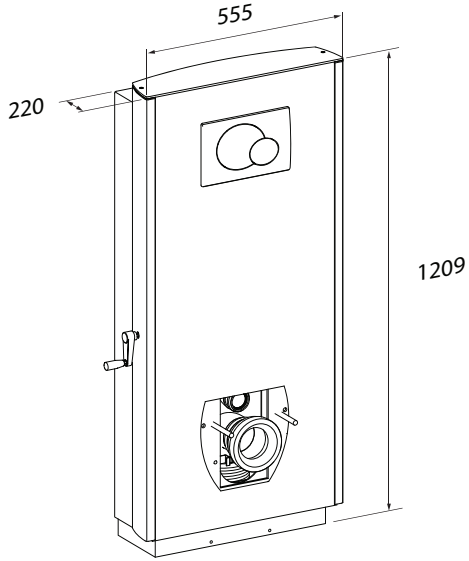
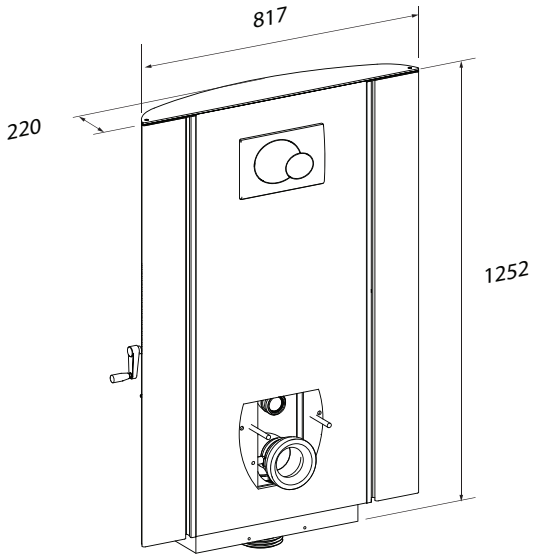
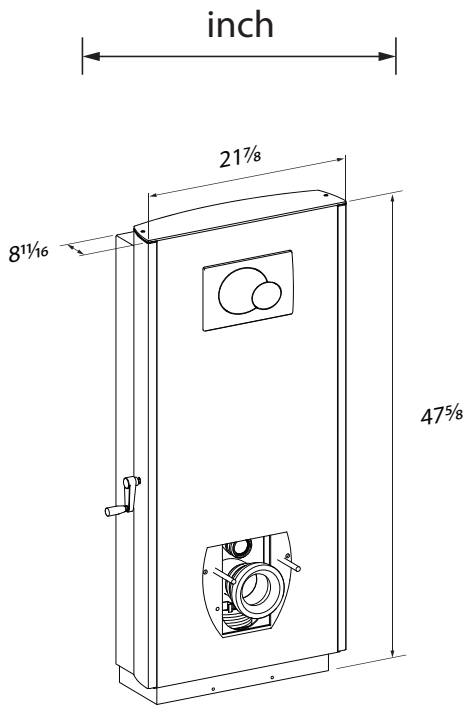
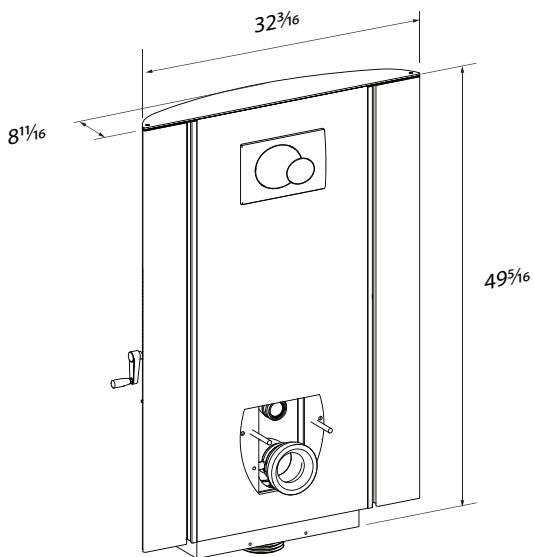
R8112, R8122, R8132, R8142

Content

Mounting instruction	8
Symbols used in this manual	9
General safety	10
Product label	11
Intended use	12
Intended operator profile	12
Operating environment	12
Operating instructions	13
Support arms (optional)	14
Cleaning	15
To remove limescale	15
Maintenance	16
Maintenance inspection	16
Technical data	17
Trouble-shooting	18
Disposal and recycling	18

en

Mounting instruction

A mounting instruction is included with the product. It can also be downloaded from www.pressalit.com.

Original instructions

Symbols used in this manual

The following symbols are used in this manual as well as the associated documentation supplied with the product.

WARNING indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

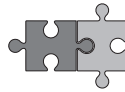
CAUTION indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury. It may also be used to alert against unsafe practices.

IMPORTANT indicates important information about handling and use of the product.

NOTICE is used to address practices not related to personal safety.

Hand injury warning Compatible products

en

General safety

WARNING

- Read and understand the manual completely before use. Keep the manual close to the unit.
- Comply with this manual to avoid accidents involving serious personal injury.
- Never use or handle this product in other ways than are specified in this manual, as this can put your safety at risk and cause damage to the product.
- Make sure that everyone who is to install or use the product and its systems has the necessary information and access to this manual.

CAUTION

- Please comply with these instructions to avoid the product becoming damaged or destroyed.
- Do not perform repairs, disassembly, or assembly operations, extensions, re-adjustments or modifications to this product. These must be carried out by Pressalit or by persons authorized by Pressalit only.
- Never allow children to operate this product unless under supervision. The product is not intended for play.
- Never use the product if it is defective or damaged. Do not use a defective product before it has been repaired.

NOTICE

- The information in this manual is based on correct fitting of the product in accordance with our Assembly instruction.
- Pressalit cannot be held liable if the product is used in any way which differs from that described in the guide.
- We reserve the right to amend this manual without prior notice.

Product label placement ①.

Intended use

This product is intended for optimum ergonomics at a toilet, for both seated and standing users with disabilities.

Intended operator profile

The product can be used by all - adults, children, young and old.

en

Operating environment

For indoor use in a wet environment, such as a bathroom or wash room. The product is intended to be used at room temperature in private homes, nursing homes and hospitals under normal working conditions.

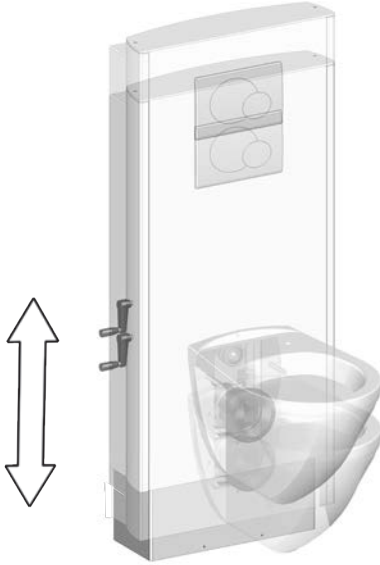
Never place objects underneath the toilet.

Operating instructions

The toilet lifter can be adjusted 300 mm / 11¹³/₁₆" up and down.

Max user weight: 200 kg / 440 lbs

Max load: 235 kg / 518 lbs

Take care to avoid injury to hands when in the vicinity of moving mechanical parts.

Lower

Raise

Turn the crank handle clockwise to raise the toilet, and counterclockwise to lower it.

Support arms (optional)

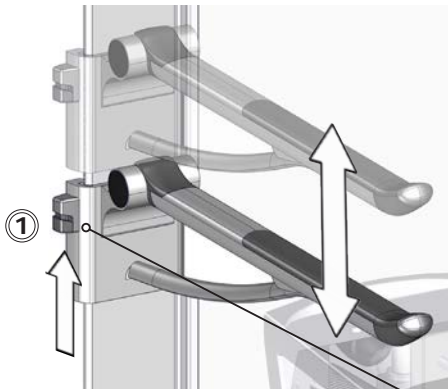
The toilet lifters model R8112, R8122, R8132 and R8142 can be equipped with support arms.

The support arms can be folded up and down. Fold them up when not in use.

The support arms can also be adjusted 240 mm / 8 7/16 inches in height.


en

To adjust upwards, pull the support arm upwards.

To adjust the support arm downwards, pinch the locking button ① and pull the support arm a little bit upwards while holding the button. Then move the support arm downwards. Release the locking button to secure the support arm in place.

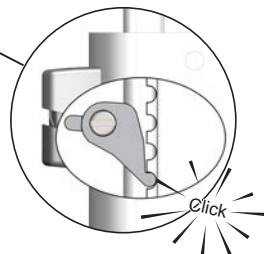Make sure that the locking button pin is engaged in the notch.

Cleaning

The product is to be cleaned with hot water or a mild soapy cleaning agent. A list of suitable cleaning agents tested by Pressalit can be found on www.pressalit.com.

CAUTION

Always read the material safety data sheet (MSDS) and supplied instructions of the cleaning, disinfecting or de-scaling agent and comply with instructions for safe use, dilution, rinsing, drying, handling and storage of the agent.

IMPORTANT

- Do not use an autoclave or other type of steam cleaner.
- Avoid aggressive cleaning agents containing acids
- Never spray the product directly with a high pressure cleaner.
- Avoid use of abrasive, corrosive or chlorine-based cleaning or disinfecting products.

To remove limescale

To remove limescale marks on the aluminium parts, use 3 parts 30% household vinegar to 7 parts water and afterwards wipe the parts with a cloth wrung out in clean water.

In countries with relatively high levels of lime in the water supply, it may be necessary to remove limescale deposits from the aluminium parts using a de-scaling agent such as acetic acid. Test this first on a area that is not visible when the product is in use.

Maintenance

Expected service life of the toilet lifter is 10 years, provided that maintenance is carried out once a year as described below.

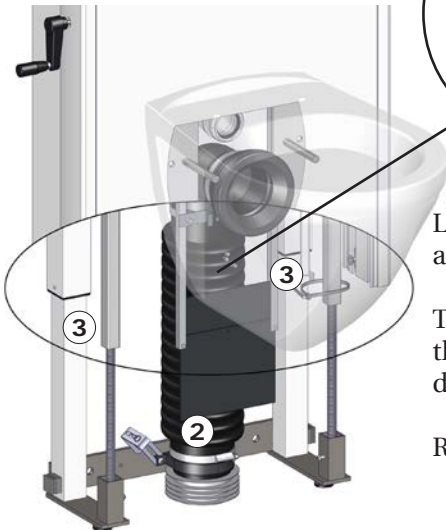
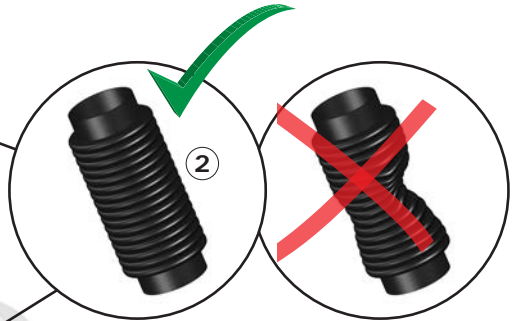
It is recommended on a monthly basis to adjust the product to its maximum positions up and down to lubricate the internal parts.

Maintenance inspection

Take the toilet to its highest position.

Remove the lower cover plate ① (can be clicked off).

Ensure that the flexible outlet hose ② can move freely when the toilet is height adjusted. The flexible outlet hose must be symmetrical when stretched out.

Lubricate the inner square tubes ③ on all 4 sides as far as can be reached.

Take the toilet to its lowest position, and then run it to the top position again to distribute the lubricant.

Reattach the lower cover plate.

Technical data

Max user weight	200 kg / 440 lbs	
Max load	235 kg / 518 lbs	
Product weight	R8012, R8022 R8032, R8042	52,3 kg / 117,50 lbs
	R8112, R8122 R8132, R8142	58,3 kg / 128,52 lbs

en

Trouble-shooting

Symptom	Cause	Action
The product runs up and down unevenly.	The inner square tubes are not properly lubricated.	Oil the inner square tubes as described under "Maintenance".
The toilet doesn't flush properly.	The flexible hose might be squeezed.	Ensure that the flexible hose is in the right position as described under "Maintenance".

Contact Pressalit A/S or your local dealer if the problem cannot be remedied. Do not attempt to repair the product yourself, as this will invalidate the guarantee.

Disposal and recycling

en

The unit contains reusable materials. There are no known hazards or risks associated with disposal of the product. All components can be discarded after having been cleaned and disinfected.

When disposing of the product, we recommend that it be disassembled and broken down into different waste groups for recycling or combustion.

The following table provides information on the recycling and handling of the product.

Recyclable components	Material
Steel	Steel can be recycled by being melted down and included as a secondary material in the production of new steel.
Plastics	Use plastic sorting and recycling systems where they are locally available, otherwise incinerating plastic has the added benefit of generating energy which can be used for, for example, combined heat and power production.

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