



Independent 4 Life

Relaxa Bath Lift Back Plate for Stud Wall Fixing

£75.60 Inc. VAT

Product Code: 4E8W027B

Product Images



Description

If fixing the Relaxa Bath Lift to a studded or non solid wall this back plate must be used.

The lifting weight of the bath lift is up to 127kg (280lbs) requiring the bath lift to be robustly secured to the wall.

When installing on a studded wall, or a wall not of a solid construction, the chassis back plate of the Relaxa Bath Lift and the wall plate must be used on the bath side.

The back plate should be used to spread the load on the rear side of the wall, and secured with screwed rods through the wall.

Care should be taken to ensure the rods finish flush with the nuts inside the unit, otherwise the belt could be damaged when the bath lift is used.

Key benefits:

- Load spreading plate for use with 'stud' or non-solid walls

Warranty:

- 1 year manufacturer warranty

Please note:

- Access to the reverse of the wall is required

Additional Information

Manufacturer

Relaxa Mobility